


THE INFORMATION CONTAINED IN THIS DRAWING IS THE SOLE PROPERTY OF ALTIMATE MEDICAL. ANY REPRODUCTION IN PART OR WHOLE WITHOUT THE WRITTEN PERMISSION OF ALTIMATE MEDICAL IS PROHIBITED.


REVISIONS				
REV:	DESCRIPTION:	ECO #:	DATE:	APPROVED:
000	NEW RELEASE.		7/12/05	JVS

R.188

4X 6.2mm THRU  
 12.6mm X 90°

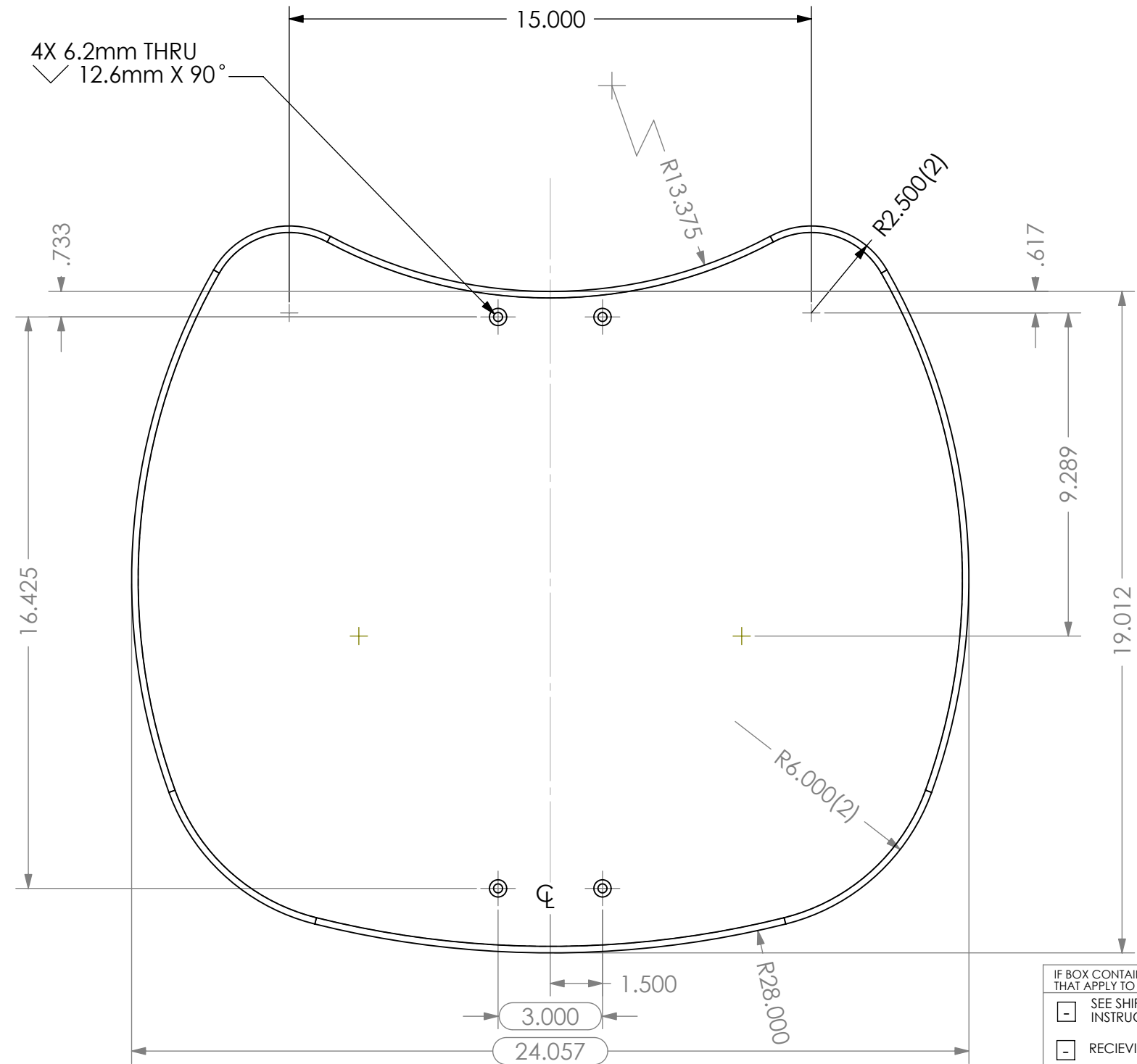

.375

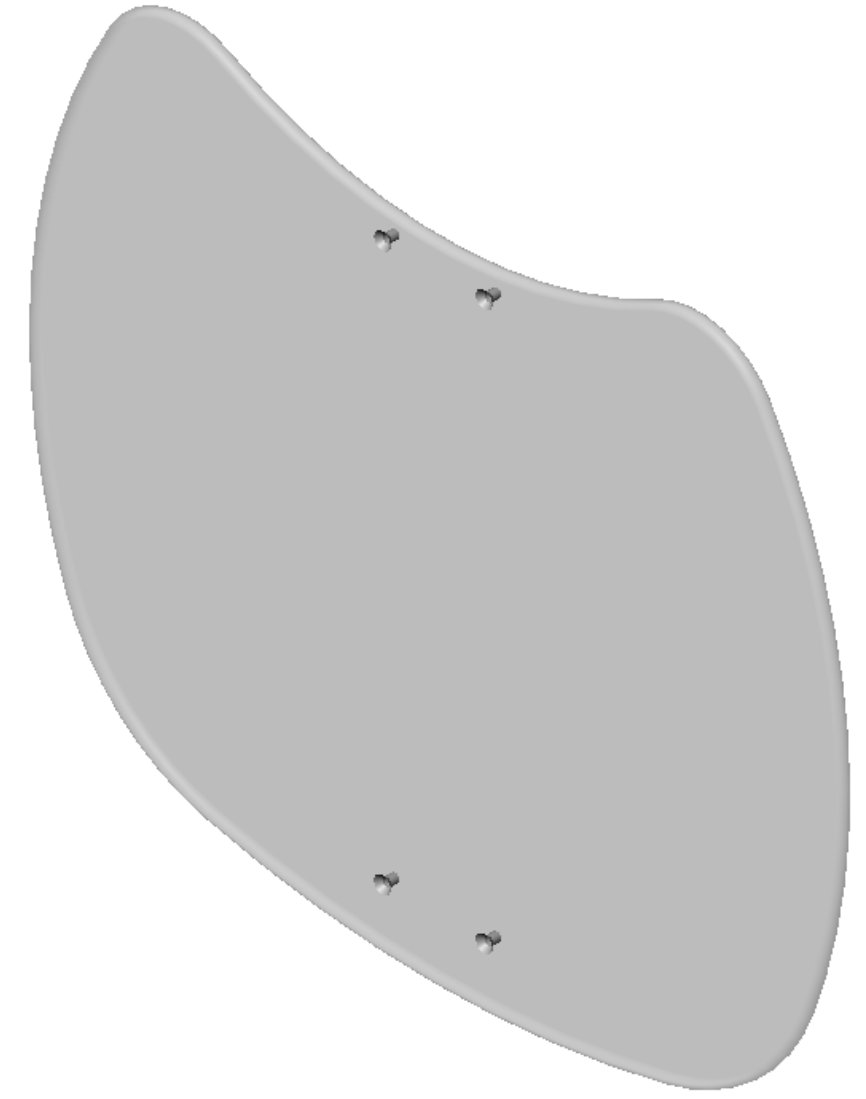

NOTE:  
 1. PART IS SYMMETRICAL ABOUT THE CENTER LINE.

- IF BOX CONTAINS AN "X", SEE NOTES THAT APPLY TO EACH SUBJECT.
- SEE SHIPPING INSTRUCTIONS
  - RECEIVING INSPECTION
  - SEE ENVIRONMENTAL INSTRUCTIONS

APPROVED:	DATE:
DRAWN: MWL	DATE: 9/15/2004
MATERIAL: CLEAR/ACRYLIC	FINISH: -
CODE: PLSTC	

UNLESS OTHERWISE SPECIFIED DIMENSIONS ARE IN INCHES. TOLERANCES AND SPECS. ARE:

DECIMALS	ANGLES
.X ± .1	.X ± .5°
.XX ± .03	
.XXX ± .015	

HOLE SIZE TOLERANCE: (+.005/-.000)  
 BREAK EDGES .005 - .015  
 CMFR DRILLED HOLES .03 X 45°

**ALTIMATE MEDICAL, INC.**  
 PO BOX 180 - 262 WEST FIRST STREET - MORTON, MN 56270  
 PHONE (507) 697-6393 FAX (507) 697-6900

DESCRIPTION: TABLE BASIC

SIZE DWG. NO. **NG20017** REV. 000

SCALE = 1:4 CAD FILE NAME: NG20017 SHEET 1 OF 1