


THE INFORMATION CONTAINED IN THIS DRAWING IS THE SOLE PROPERTY OF ALTIMATE MEDICAL. ANY REPRODUCTION IN PART OR WHOLE WITHOUT THE WRITTEN PERMISSION OF ALTIMATE MEDICAL IS PROHIBITED.

REVISIONS				
REV:	DESCRIPTION:	ECO #:	DATE:	APPROVED:
000	NEW RELEASE.	05-ECO-001	8/12/05	GJR
A	CHANGED HOLE PLACEMENT OF NG20058 FROM 6" TO 5.75"	05-ECO-018	11-2-05	GJR

- NOTES:
 1. STAPLE COVER AND FELT TO PLASTCI BOARD USING BLACK STAPLES.
 2. ATTACH COVER WITH POLYURETHANE COATING FACING OUTWARD.

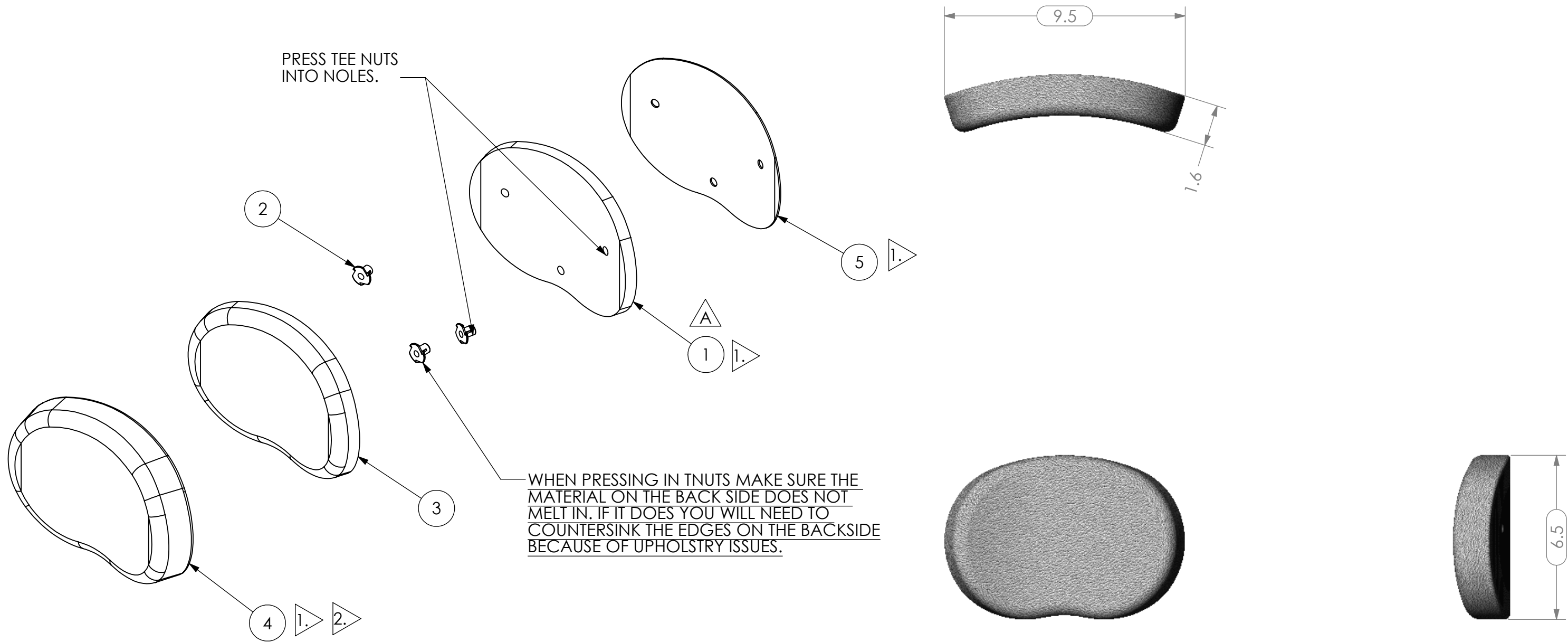

ITEM NO.	PART NO.	REV	QTY.	NAME
1	NG20058	A	1	BOARD PAD CHEST STANDARD
2	600177	000	3	TEE NUT M8 X 1.25
3	NG30085	000	1	CUSHION PAD CHEST STANDARD
4	NG30086	000	1	COVER PAD CHEST STANDARD
5	NG30087	000	1	FELT PAD CHEST STANDARD

IF BOX CONTAINS AN "X", SEE NOTES THAT APPLY TO EACH SUBJECT.		APPROVED:	DATE:	UNLESS OTHERWISE SPECIFIED DIMENSIONS ARE IN INCHES. TOLERANCES AND SPECS. ARE: DECIMALS: .X ± .1 ANGLES: .X ± .5° .XX ± .03 .XXX ± .015 HOLE SIZE TOLERANCE: (+.005/-.000) BREAK EDGES .005 - .015 CMFR DRILLED HOLES .03 X 45°	ALTIMATE MEDICAL, INC. PO BOX 180 - 262 WEST FIRST STREET - MORTON, MN 56270 PHONE (507) 697-6393 FAX (507) 697-6900 DESCRIPTION: PAD CHEST STANDARD SIZE DWG. NO. NG30001 REV. A SCALE = 1:4 CAD FILE NAME: NG30001 SHEET 1 OF 1	
<input type="checkbox"/> SEE SHIPPING INSTRUCTIONS	<input type="checkbox"/> RECEIVING INSPECTION	DRAWN: MWL	DATE: 8/12/05			MATERIAL:
<input type="checkbox"/> SEE ENVIRONMENTAL INSTRUCTIONS		OPTION NUMBER:				FINISH: BLACK
						CODE: UPSTRY