


THE INFORMATION CONTAINED IN THIS DRAWING IS THE SOLE PROPERTY OF  
 ALTIMATE MEDICAL. ANY REPRODUCTION IN PART OR WHOLE WITHOUT  
 THE WRITTEN PERMISSION OF ALTIMATE MEDICAL IS PROHIBITED.

REVISIONS				
REV:	DESCRIPTION:	ECO #:	DATE:	APPROVED:
000	NEW RELEASE.	DCR-00109	1/9/07	MWL
A	REVISED ITEM NG50276.	DCR-00160	3/19/07	MWL
B	REVISED ITEMS NG50274 AND NG50275. REMOVED NG50276.	DCR-00244	8/7/07	MWL
C	REVISED NG10749 TO REVISION A.	DCR-00289	11/14/07	MWL
D	REVISED NG10749 TO REV B	DCR-00369	3/19/08	JVS
E	REVISED NG10755 TO A REV A	DCR-00543	6/4/09	JVS
F	REVISED NG50274 AND NG50275.	DCR-00550	6/11/09	MWL
G	REVISED NG50274 AND NG50275 TO A REV C	DCR-00673	5/11/10	JVS
H	REVISED NG50274 AND NG50275 TO A REV D	DCR-00676	5/24/10	MWL
I	REVISED NG50274 AND NG50275 TO A REV E	DCR-00753	12/30/10	MWL

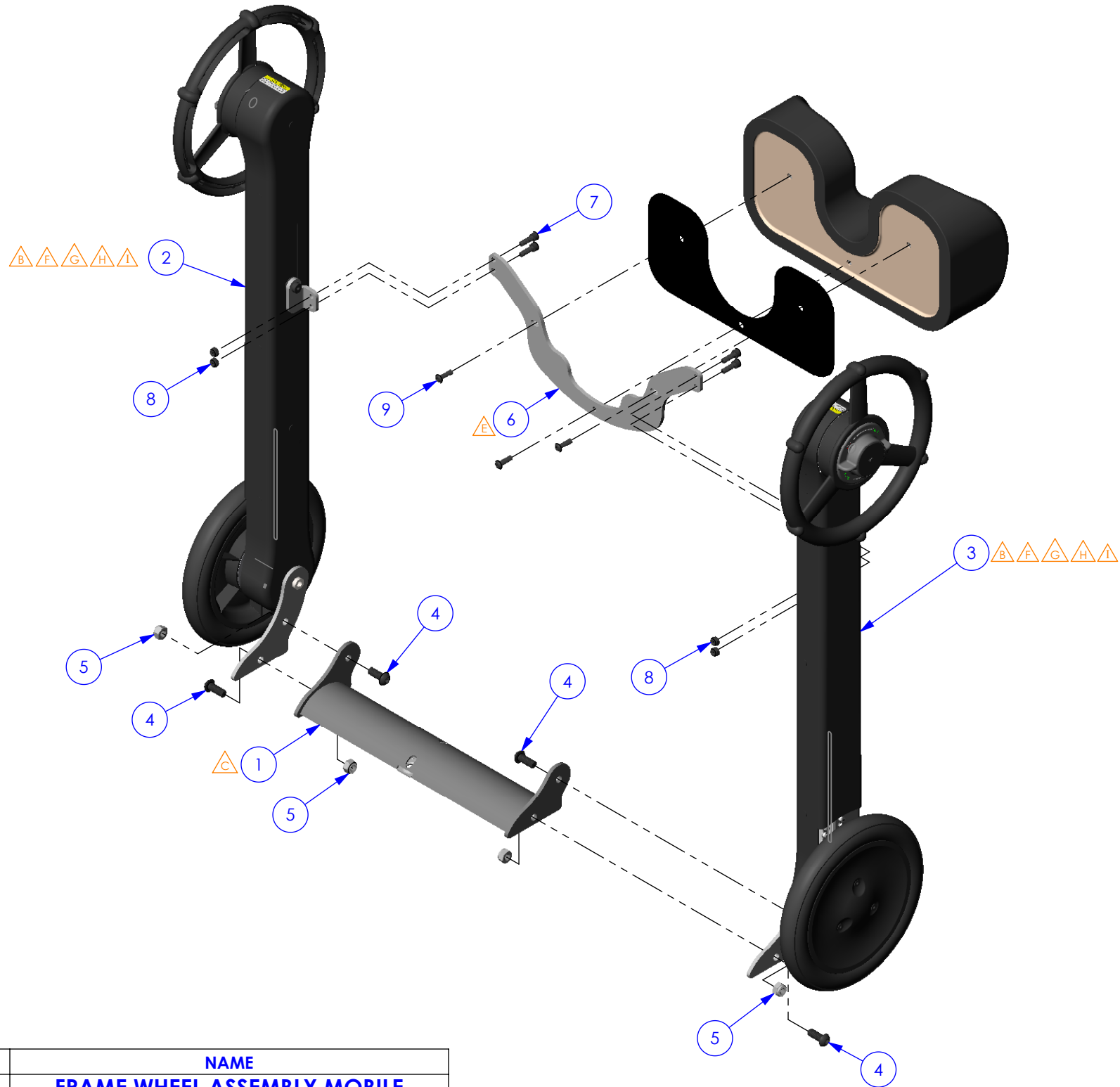

ITEM NO.	PART NO.	REV	QTY.	NAME
1	NG10749	B	1	FRAME WHEEL ASSEMBLY MOBILE
2	NG50274	E	1	ASMBLY MOBILE RH
3	NG50275	E	1	ASMBLY MOBILE LH
4	600274	000	4	BOLT M12 X 1.75 - 30 BTN SHCS
5	600011	000	4	NUT M12 X 1.75 NYLON LOCK
6	NG10755	A	1	PLATE BRACKET MOBILE MOUNT
7	600035	000	4	BOLT M8 X 1.25 - 25 SHCS
8	600009	000	4	NUT M8 X 1.25 NYLON LOCK
9	600133	000	3	BOLT M8 X 25 BTN SHCS

IF BOX CONTAINS AN "X", SEE NOTES THAT APPLY TO EACH SUBJECT.

- SEE SHIPPING INSTRUCTIONS
- RECEIVING INSPECTION
- SEE ENVIRONMENTAL INSTRUCTIONS

APPROVED: \_\_\_\_\_ DATE: \_\_\_\_\_

DRAWN: MWL DATE: 2/28/06 MATERIAL: \_\_\_\_\_

OPTION NUMBER: -

FINISH: -  
 CODE: ASMBLY

UNLESS OTHERWISE SPECIFIED DIMENSIONS ARE IN INCHES. TOLERANCES AND SPECS. ARE:

DECIMALS .X ± .1  
 .XX ± .03  
 .XXX ± .015  
 ANGLES .X ± .5°  
 HOLE SIZE TOLERANCE: (+.005/-.000)  
 BREAK EDGES .005 - .015  
 CMFR DRILLED HOLES .03 X 45°

ALTIMATE MEDICAL, INC.

PO BOX 180 - 262 WEST FIRST STREET - MORTON, MN 56270  
 PHONE (507) 697-6393 FAX (507) 697-6900

DESCRIPTION: MOBILE OPTION EVOLV

SIZE DWG. NO. NG50025 REV. I

SCALE = 1:8 CAD FILE NAME: NG50025 SHEET 1 OF 1