


THE INFORMATION CONTAINED IN THIS DRAWING IS THE SOLE PROPERTY OF
 ALTIMATE MEDICAL. ANY REPRODUCTION IN PART OR WHOLE WITHOUT
 THE WRITTEN PERMISSION OF ALTIMATE MEDICAL IS PROHIBITED.

REVISIONS				
REV:	DESCRIPTION:	ECO #:	DATE:	APPROVED:
000	NEW RELEASE.	-	7/12/05	JVS
A	UPDATE D PART #NG10244 TO A REV B	DCR-00151	3/6/07	JVS

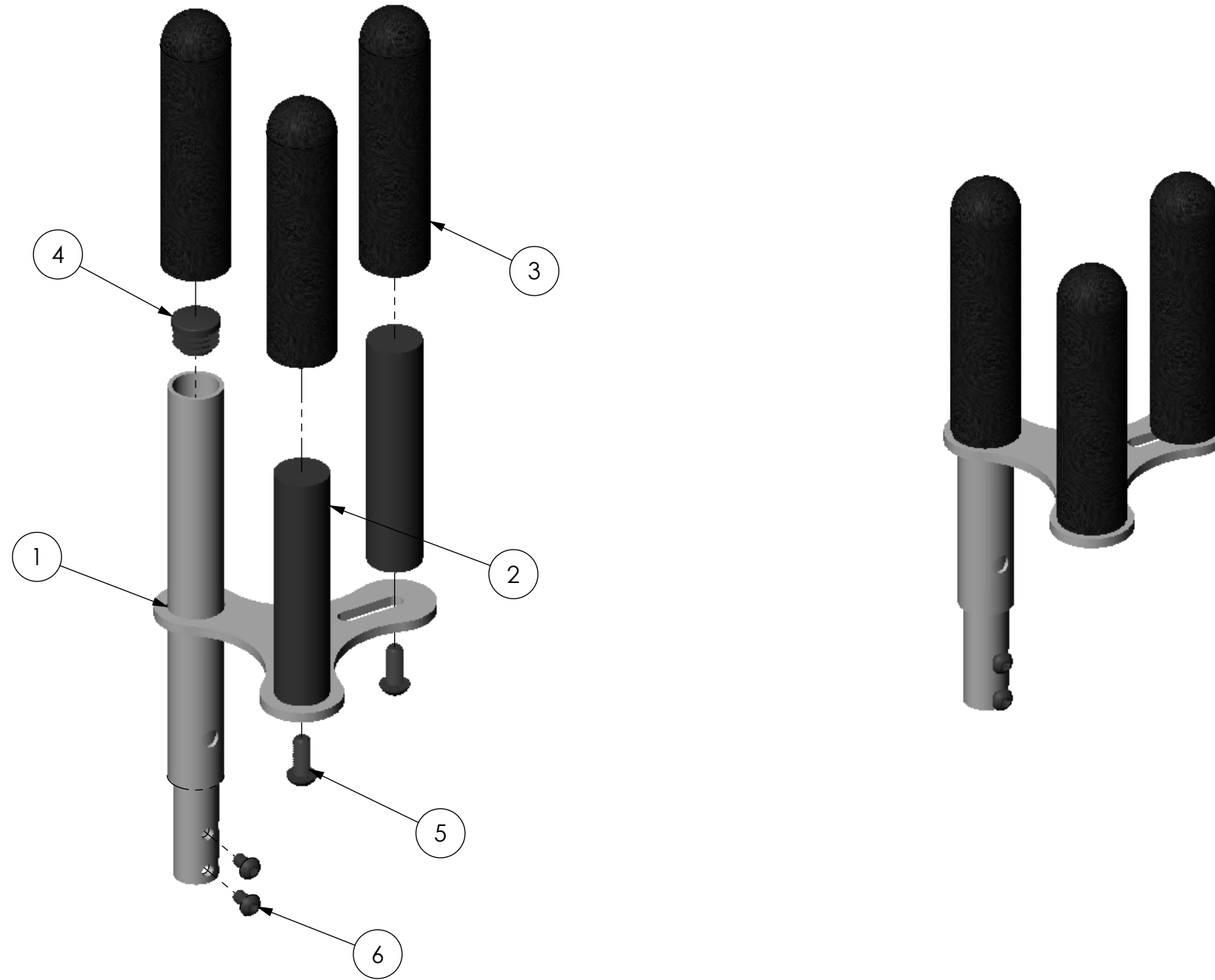

ITEM NO.	PART NO.	REV	QTY.	NAME
1	NG10244	B	1	BRACKET QUAD GRIP
2	ng20051	000	2	SHAFT SUPPORT GRIP
3	600100	000	3	GRIP DIPPED FOAM 1.00 ID
4	10764	00	1	CAP
5	600053	000	2	BOLT M8 X 20 BTN SHCS
6	600037	000	2	BOLT BTNHD M6 X 1.5-10

IF BOX CONTAINS AN "X", SEE NOTES THAT APPLY TO EACH SUBJECT. <input type="checkbox"/> SEE SHIPPING INSTRUCTIONS <input type="checkbox"/> RECEIVING INSPECTION <input type="checkbox"/> SEE ENVIRONMENTAL INSTRUCTIONS	APPROVED:	DATE:	UNLESS OTHERWISE SPECIFIED DIMENSIONS ARE IN INCHES. TOLERANCES AND SPECS. ARE: DECIMALS: .X ± .1 .XX ± .03 .XXX ± .015 ANGLES: .X ± .5° HOLE SIZE TOLERANCE: (+.005/- .000) BREAK EDGES .005 - .015 CMFR DRILLED HOLES .03 X 45°	ALTIMATE MEDICAL, INC. PO BOX 180 - 262 WEST FIRST STREET - MORTON, MN 56270 PHONE (507) 697-6393 FAX (507) 697-6900 DESCRIPTION: QUAD GRIP OPTION SIZE DWG. NO. NG50044 REV. A SCALE = 1:2.5 CAD FILE NAME: NG50044 SHEET 1 OF 1	
	DRAWN: GJR	DATE: 3/7/05			MATERIAL:
	OPTION NUMBER: V602	FINISH: -			CODE: ASMBLY