


THE INFORMATION CONTAINED IN THIS DRAWING IS THE SOLE PROPERTY OF
 ALTIMATE MEDICAL. ANY REPRODUCTION IN PART OR WHOLE WITHOUT
 THE WRITTEN PERMISSION OF ALTIMATE MEDICAL IS PROHIBITED.

REVISIONS				
REV:	DESCRIPTION:	ECO #:	DATE:	APPROVED:
000	NEW RELEASE	05-ECO-006	10/5/05	MWL
A	REVISED ITEM NG10371.	DCR-00161	3/20/07	MWL
B	REVISED SOCKET ANGLE ON NG10371.	DCR-00471	12/12/08	MWL
C	REVISED NG10371 TO REV C, REPLACED 91501 WITH 91406 & 91511	DCR-00570	7/29/09	JVS

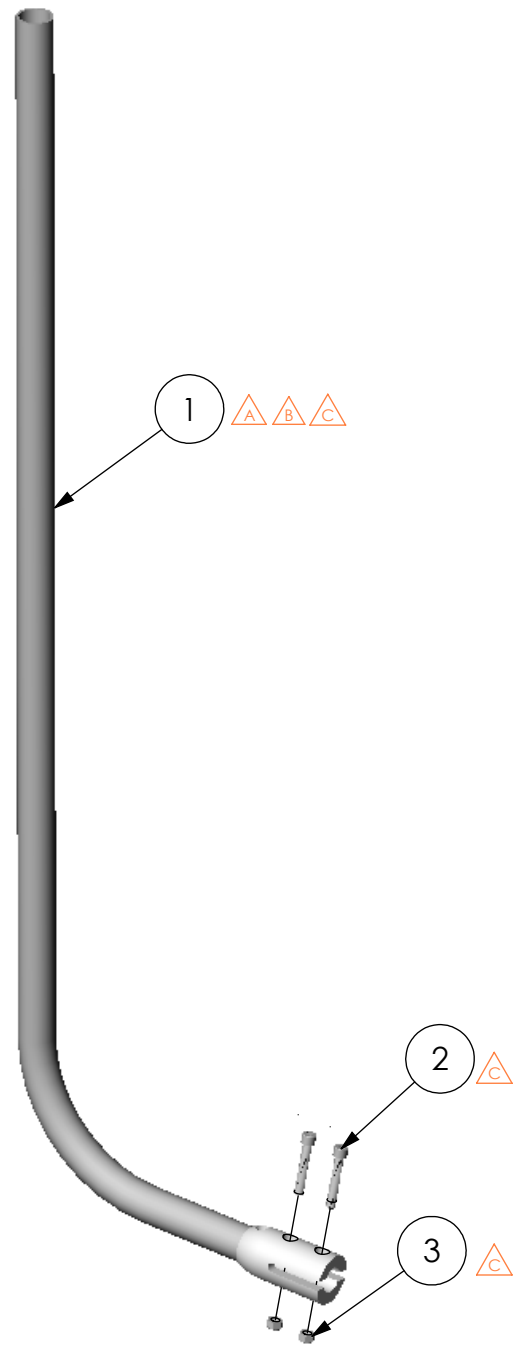

ITEM NO.	PART NO.	REV	QTY.	NAME
1	NG10371	C	1	PUMP HANDLE WELDMENT LH
2	91406	000	2	BOLT 1/4-20 X 1-1/2 SHCS
3	91511	000	2	NUT 1/4-20 NYLON LOCK

IF BOX CONTAINS AN "X", SEE NOTES THAT APPLY TO EACH SUBJECT. <input type="checkbox"/> SEE SHIPPING INSTRUCTIONS <input type="checkbox"/> RECEIVING INSPECTION <input type="checkbox"/> SEE ENVIRONMENTAL INSTRUCTIONS	APPROVED:	DATE:	UNLESS OTHERWISE SPECIFIED DIMENSIONS ARE IN INCHES. TOLERANCES AND SPECS. ARE: DECIMALS: .X ± .1 .XX ± .03 .XXX ± .015 HOLE SIZE TOLERANCE: (+.005/- .000) BREAK EDGES .005 - .015 CMFR DRILLED HOLES .03 X 45°	ALTIMATE MEDICAL, INC. PO BOX 180 - 262 WEST FIRST STREET - MORTON, MN 56270 PHONE (507) 697-6393 FAX (507) 697-6900 DESCRIPTION: HANDLE PUMP STANDARD LH SIZE DWG. NO. NG50126 REV. C SCALE = 1:5 CAD FILE NAME: NG50126 SHEET 1 OF 1
	DRAWN: MWL	DATE: 10/5/05		
			FINISH: -	
			CODE: ASMBLY	