
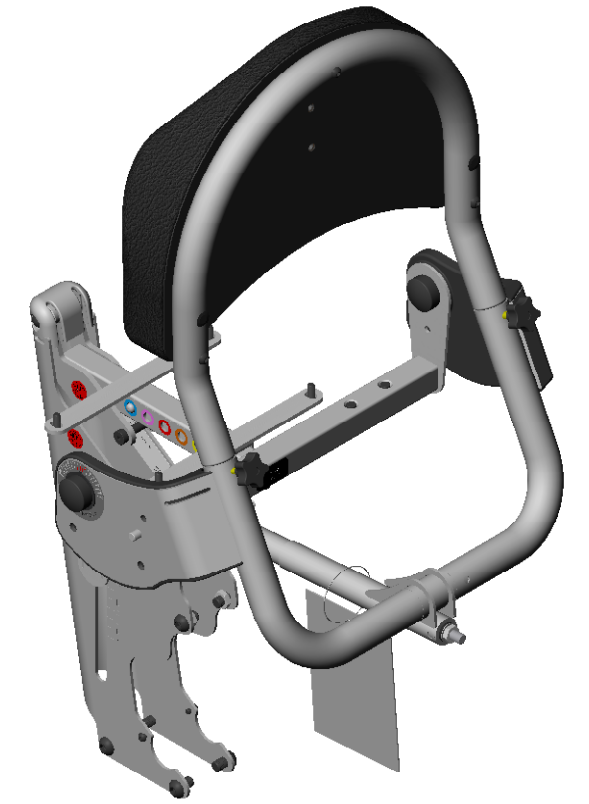


THE INFORMATION CONTAINED IN THIS DRAWING IS THE SOLE PROPERTY OF ALTIMATE MEDICAL. ANY REPRODUCTION IN PART OR WHOLE WITHOUT THE WRITTEN PERMISSION OF ALTIMATE MEDICAL IS PROHIBITED.

| REVISIONS | | | | |
|-----------|--------------------------------|------------|----------|-----------|
| REV: | DESCRIPTION: | ECO #: | DATE: | APPROVED: |
| 000 | NEW RELEASE. | 06-ECO-001 | 1/23/06 | MWL |
| A | REPLACED NG50086 WITH NG50165. | 06-ECO-051 | 11/21/06 | MWL |
| B | REVISED NG50165 TO REVISION A. | DCR-00111 | 1/11/07 | MWL |
| C | REVISED NG50165 TO REVISION B. | DCR-00243 | 8/6/07 | MWL |
| D | REVISED NG50165 TO REVISION C. | DCR-00291 | 11/15/07 | MWL |
| E | REVISED NG50165 TO REVISION D. | DCR-00395 | 4/23/08 | MWL |
| F | REORGANIZED BILL OF MATERIALS. | DCR-00459 | 10/27/08 | MWL |

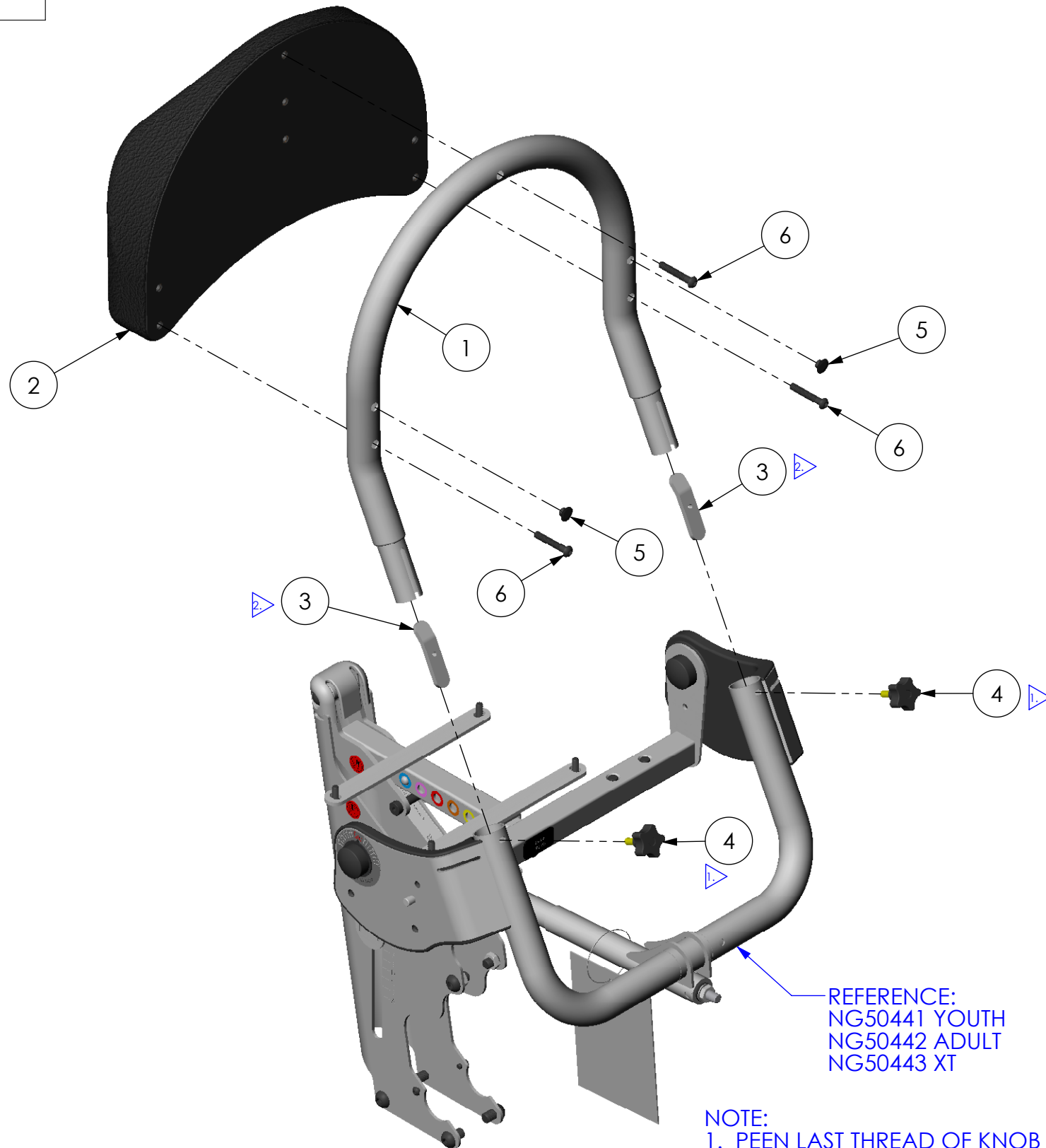

REFERENCE:
 NG50441 YOUTH
 NG50442 ADULT
 NG50443 XT

NOTE:
 1. PEEN LAST THREAD OF KNOB (600054) AFTER THREADING ON TAB (NG10260).
 2. NOTE ORIENTATION OF TAB.

| ITEM NO. | PART NO. | REV | QTY. | NAME |
|----------|----------|-----|------|-------------------------------|
| 1 | NG10474 | 000 | 1 | TUBE REMOVABLE BACK REST HIGH |
| 2 | NG30049 | 000 | 1 | PAD BACK REST CONTOURED HIGH |
| 3 | NG10260 | A | 2 | TAB INNER REMOVEABLE HANDLE |
| 4 | 600054 | 000 | 2 | KNOB M6 X 3/4 ZC 5 PT |
| 5 | 600255 | 000 | 2 | CAP END M8 FLUSH |
| 6 | 600060 | 000 | 3 | BOLT M8 X 1.25 X 50 BTN |

| | | | | |
|---|---|-------|--|--|
| IF BOX CONTAINS AN "X", SEE NOTES THAT APPLY TO EACH SUBJECT. <input type="checkbox"/> SEE SHIPPING INSTRUCTIONS <input type="checkbox"/> RECEIVING INSPECTION <input type="checkbox"/> SEE ENVIRONMENTAL INSTRUCTIONS | APPROVED: | DATE: | UNLESS OTHERWISE SPECIFIED DIMENSIONS ARE IN INCHES. TOLERANCES AND SPECS. ARE: DECIMALS ANGLES .X ± .1 .X ± .5° .XX ± .03 .XXX ± .015 HOLE SIZE TOLERANCE: (+.005/-.000) BREAK EDGES .005 - .015 CMFR DRILLED HOLES .03 X 45° | ALTIMATE MEDICAL, INC. PO BOX 180 - 262 WEST FIRST STREET - MORTON, MN 56270 PHONE (507) 697-6393 FAX (507) 697-6900 DESCRIPTION: BACK REMOVABLE HIGH CONTOURED SIZE DWG. NO. NG50174 REV. F SCALE = 1:6 CAD FILE NAME: NG50174 SHEET 1 OF 1 |
| | DRAWN: MWL DATE: 1/23/06 MATERIAL: FINISH: EXPIRATION DATE: CODE: ASMBLY | | | |