


THE INFORMATION CONTAINED IN THIS DRAWING IS THE SOLE PROPERTY OF ALTIMATE MEDICAL. ANY REPRODUCTION IN PART OR WHOLE WITHOUT THE WRITTEN PERMISSION OF ALTIMATE MEDICAL IS PROHIBITED.

REVISIONS				
REV:	DESCRIPTION:	ECO #:	DATE:	APPROVED:
000	NEW RELEASE.	05-ECO-001	8/22/05	GJR
A	REVISED PARTS NG11045, NG11053 & NG11054 TO REV A	DCR-00334	1/31/08	JVS
B	REVISED NG11050. REPLACED NG10313 WITH NG20125.	DCR-00436	6/18/08	MWL
B	ADDED WARNING STICKER 80890 AND NOTE #2.	DCR-00436	6/18/08	MWL
C	REVISED NG20125.	DCR-00467	11/3/08	MWL
D	REPLACING NG30052L & R WITH NG30289L & R/ADDING NG20134	DCR-00750	12/16/10	JVS
E	SWITCHED #80890 WITH Y4001, NEW PINCH STICKER	DCR-00919	4/26/12	JVS

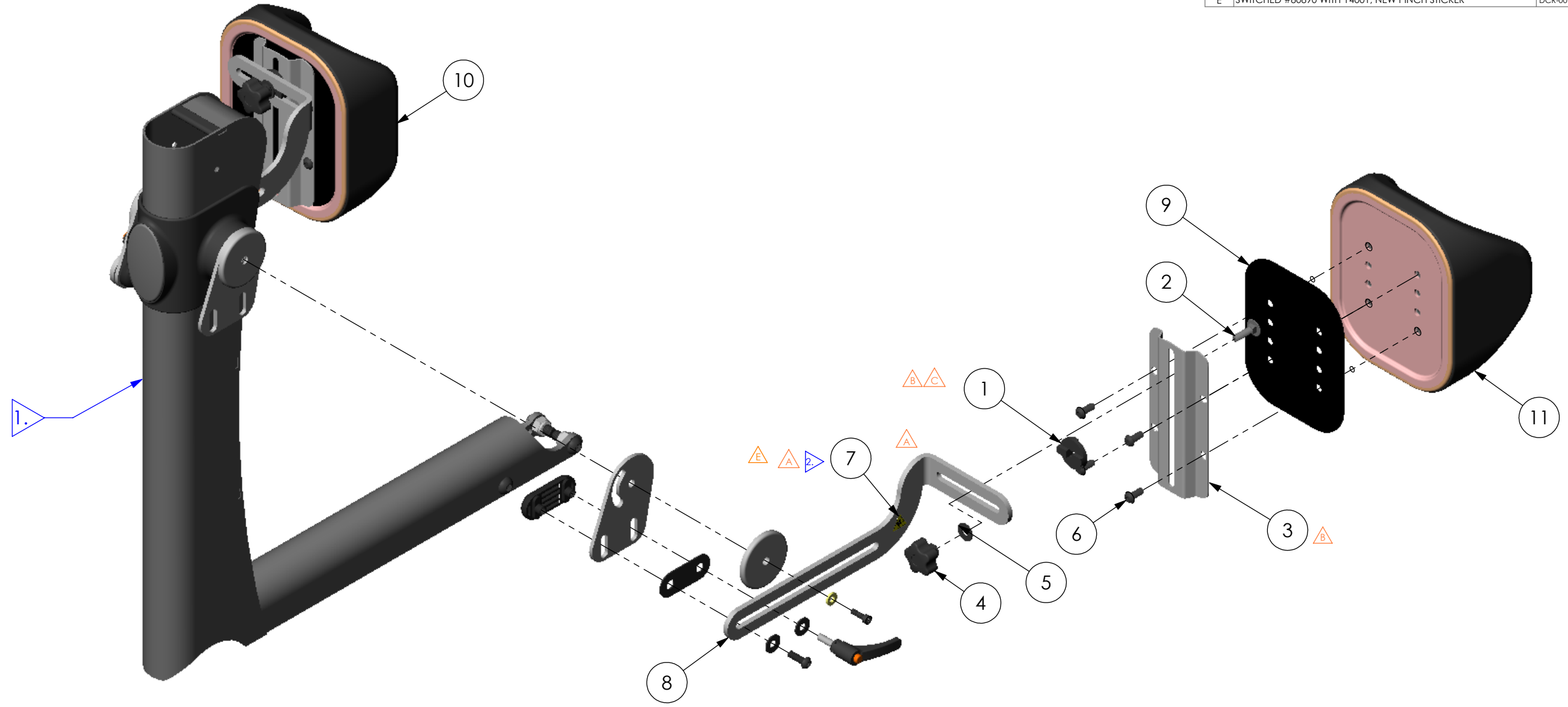

ITEM NO.	PART NO.	REV	Default/QT Y.	NAME
1	NG20125	A	2	LUG ADJUSTMENT GUIDE
2	600091	000	2	BOLT M8 X 1.25 - 30 CARRIAGE
3	NG11050	A	2	BRACKET EVOLV IND KNEE PAD BRACKET
4	600132	000	2	KNOB M8 X 1.25 5 POINT FEMALE
5	91603	01	2	FLAT WASHER
6	600053	000	8	BOLT M8 X 20 BTN SHCS
7	Y4001	000	2	LABEL HAND CRUSH
8	NG50423	000	1	KNEE BRACKET ASSEMBLY
9	NG20134	000	2	KNEE COVER BACK INDEPENDENT
10	NG30289R	000	1	KNEE PAD ASSY INDEPENDENT RIGHT
11	NG30289L	000	1	KNEE PAD ASSY INDEPENDENT LEFT

NOTE:
 1. REFERENCE NG50376 FOR YOUTH AND ADULT AND NG50377 FOR XT.
 2. APPLY WARNING LABEL ON BOTH SIDES.

IF BOX CONTAINS AN "X", SEE NOTES THAT APPLY TO EACH SUBJECT.		APPROVED: _____ DATE: _____		UNLESS OTHERWISE SPECIFIED DIMENSIONS ARE IN INCHES. TOLERANCES AND SPECS. ARE: DECIMALS .X ± .1 .XX ± .03 .XXX ± .015 HOLE SIZE TOLERANCE: (+.005/- .000) BREAK EDGES .005 - .015 CMFR DRILLED HOLES .03 X 45°	ALTIMATE MEDICAL, INC. PO BOX 180 - 262 WEST FIRST STREET - MORTON, MN 56270 PHONE (507) 697-6393 FAX (507) 697-6900 DESCRIPTION: INDEPENDENT KNEES EVOLV SIZE DWG. NO. NG50384 REV. E SCALE = 1:5 CAD FILE NAME: NG50384 SHEET 1 OF 1
<input type="checkbox"/> SEE SHIPPING INSTRUCTIONS	<input type="checkbox"/> RECEIVING INSPECTION	DRAWN: JVS	DATE: 1/24/08		
<input type="checkbox"/> SEE ENVIRONMENTAL INSTRUCTIONS		MATERIAL:	FINISH: -		
		CODE: ASMBLY			