
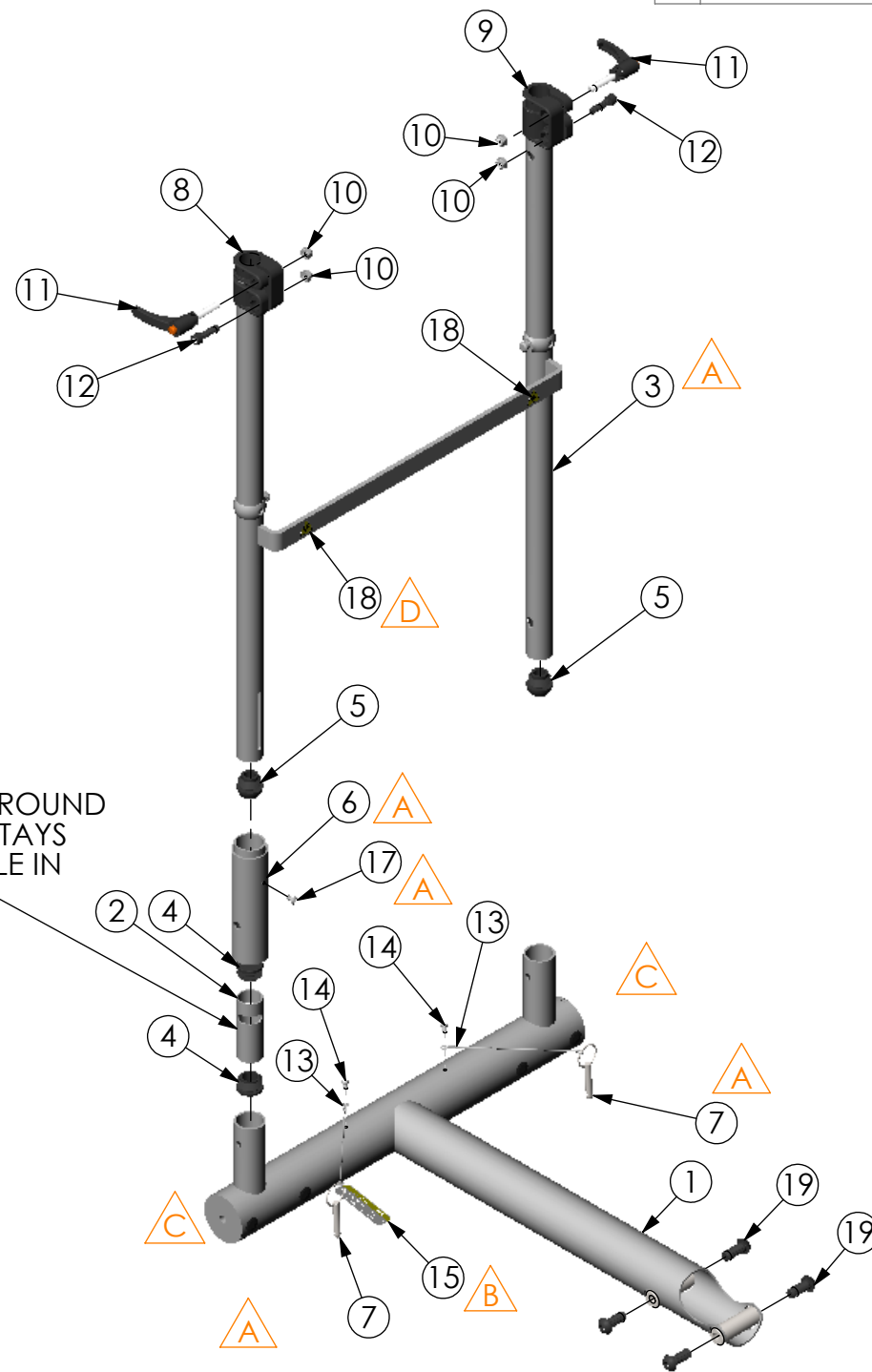
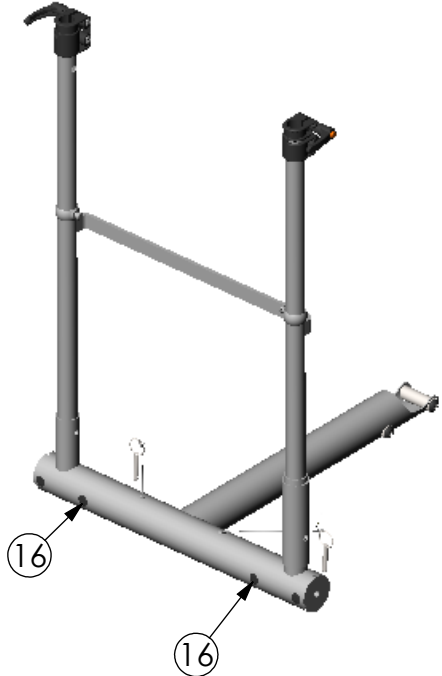


THE INFORMATION CONTAINED IN THIS DRAWING IS THE SOLE PROPERTY OF ALTIMATE MEDICAL. ANY REPRODUCTION IN PART OR WHOLE WITHOUT THE WRITTEN PERMISSION OF ALTIMATE MEDICAL IS PROHIBITED.

REVISIONS				
REV:	DESCRIPTION:	ECO #:	DATE:	APPROVED:
000	NEW RELEASE	DCR-00475	12/23/08	JVS
A	REVISED NG11097 & NG11080 TO REV A, ALSO CHANGED 600332	DCR-00487	2/2/09	JVS
-	TO 91603 AND 600389 TO 600307.	-	-	-
B	REVISED #NG40045 TO REV A	DCR-00706	9/14/10	JVS
C	REMOVED 600006, 600147, 600218, AND 91603	DCR-00744	12/9/10	TJF
D	SWITCHED #80890 WITH Y4001, NEW PINCH STICKER	DCR-00919	4/24/12	JVS

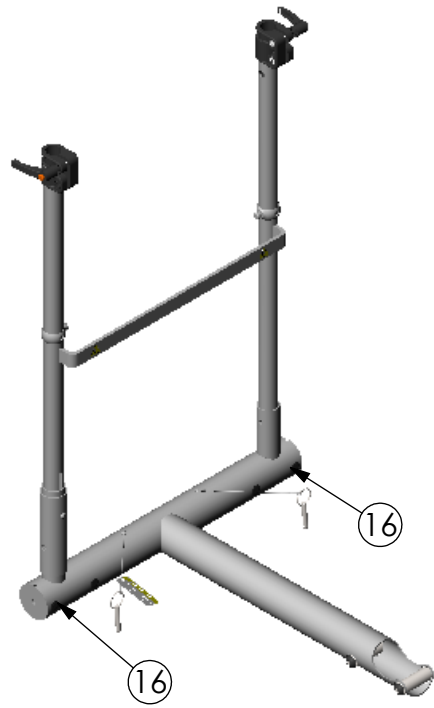

ITEM NO.	PART NO.	REV	QTY.	NAME
1	NG11061	000	1	FRONT FRAME SWING OUT H STYLE
2	NG11092	A	1	FRAME H PLUG
3	NG11097	A	1	FRAME-H LOWER WELDMENT RH
4	NG20105	000	2	CAP
5	600082	000	2	BALL CAP 1.25D 14-23 GA
6	NG11080	A	1	SWING AWAY SLEEVE EVOLV
7	92602	000	2	QUICK PIN
8	NG20120	000	1	CLAMP VERTICAL EVOLV LH
9	NG20121	000	1	CLAMP VERTICAL EVOLV RH
10	600009	000	4	NUT LOCK M8 X 1.25
11	600156	000	2	CLAMP LEVER 8mm X 40mm
12	600139	000	2	BOLT SHCS M8 X 1.25-30
13	600395	000	2	LANYARD
14	600394	000	2	SCREW SELF-TAP #8-18 X 1/2" PH PAN HD TYPE
15	NG40045	A	1	CAUTION TAG
16	600255	000	8	CAP END M8 FLUSH
17	600307	000	1	BOLT BTNHD M5 X .8-14
18	Y4001	000	2	LABEL HAND CRUSH
19	600274	000	4	BOLT M12 X 1.75 - 30 BTN SHCS

NOTE: WHEEL OPTIONS  
 NG50488 - STANDARD WHEEL  
 NG50483 - SWIVEL WHEEL

IF BOX CONTAINS AN "X", SEE NOTES THAT APPLY TO EACH SUBJECT. <input type="checkbox"/> SEE SHIPPING INSTRUCTIONS <input type="checkbox"/> RECEIVING INSPECTION <input type="checkbox"/> SEE ENVIRONMENTAL INSTRUCTIONS	APPROVED:	DATE:	UNLESS OTHERWISE SPECIFIED DIMENSIONS ARE IN INCHES. TOLERANCES AND SPECS. ARE: DECIMALS: .X ± .1 .XX ± .03 .XXX ± .015 HOLE SIZE TOLERANCE: (+.005/- .000) BREAK EDGES .005 - .015 CMFR DRILLED HOLES .03 X 45°	<b>ALTIMATE MEDICAL, INC.</b> PO BOX 180 - 262 WEST FIRST STREET - MORTON, MN 56270 PHONE (507) 697-6393 FAX (507) 697-6900 DESCRIPTION: EVOLV H-FRAME RH SIZE DWG. NO. <b>NG50417</b> REV. <b>D</b> SCALE = 1:8.5 CAD FILE NAME: NG50417 SHEET 1 OF 1	
	DRAWN: JVS	DATE: 7/16/08			MATERIAL:
	EXPIRATION DATE:				FINISH: -