


THE INFORMATION CONTAINED IN THIS DRAWING IS THE SOLE PROPERTY OF ALTIMATE MEDICAL. ANY REPRODUCTION IN PART OR WHOLE WITHOUT THE WRITTEN PERMISSION OF ALTIMATE MEDICAL IS PROHIBITED.

REVISIONS				
REV.	DESCRIPTION	ECO #	DATE	APPROVED
000	NEW RELEASE	DCR 00459	10/27/08	MWL
A	REPLACED NG10410 WITH NG11231	DCR 00766	2/17/11	TJF

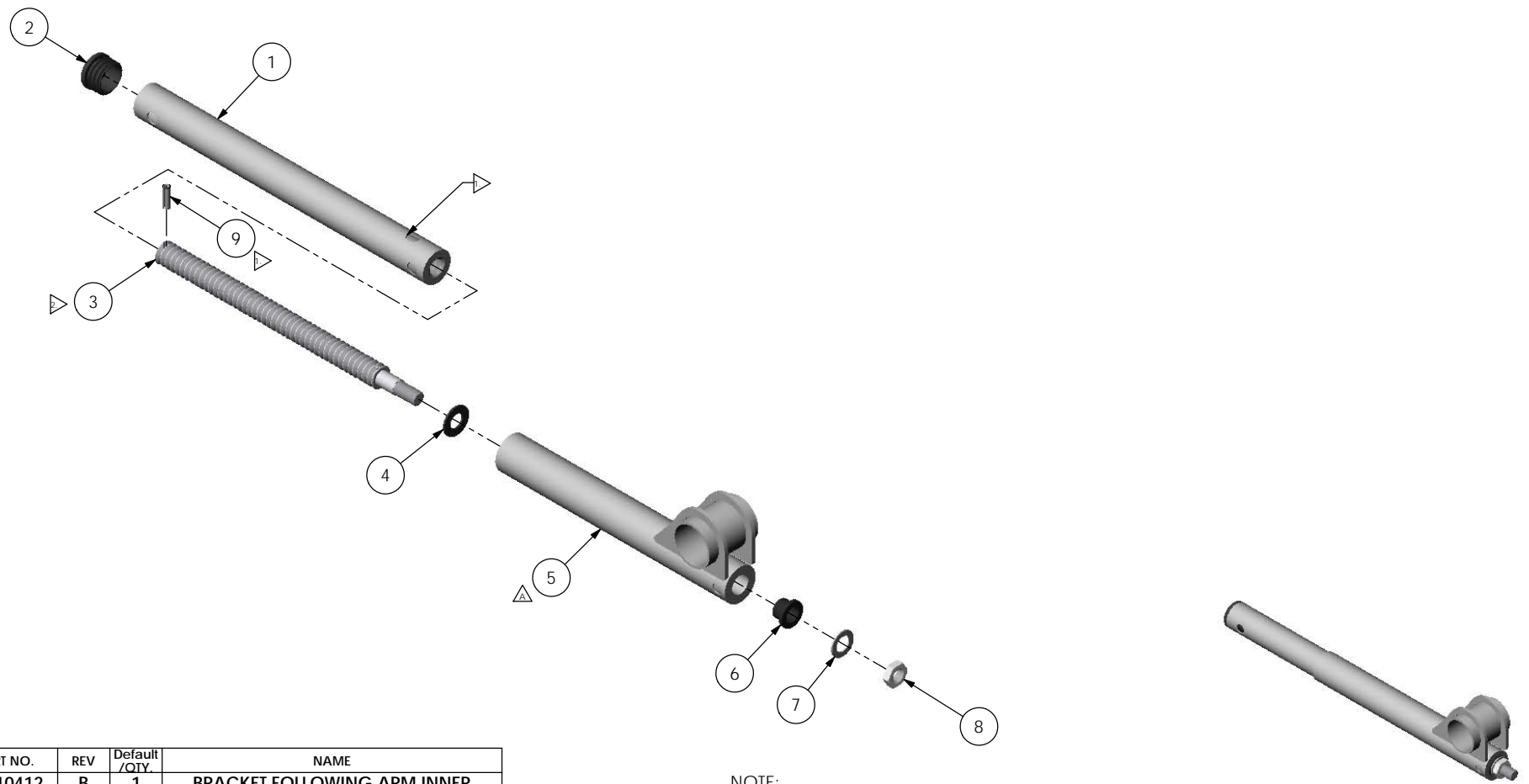

ITEM NO.	PART NO.	REV	Default /QTY	NAME
1	NG10412	B	1	BRACKET FOLLOWING ARM INNER
2	600217	000	1	CAP END RND 1.125 OD
3	600004	B	1	SHAFT ADJ. FOLLOWING ARM
4	80283	-	1	FLAT WASHER
5	NG11231	000	1	BRACKET REAR FOLLOWING ARM YOUTH
6	81256	000	1	WASHER-SHOULDER, NYLON
7	91602	-	1	BUSHING (SHIM)
8	600059	000	1	NUT JAM M12 X 1.75
9	600411	000	1	PIN SPRING 5mm X 20mm

NOTE:
 1. INSTALL PIN THROUGH ACCESS HOLE.
 2. APPLY ANTI-SEIZE LUBRICANT (600189) TO THREADS PRIOR TO ASSEMBLY. CHECK EXP. DATE.

<input type="checkbox"/> SEE SHIPPING INSTRUCTIONS <input type="checkbox"/> RECEIVING INSPECTION <input type="checkbox"/> SEE ENVIRONMENTAL INSTRUCTIONS		APPROVED: _____ DATE: _____ DRAWN: MWL DATE: 9/25/08 MATERIAL: _____ FINISH: _____ EXPIRATION DATE: _____ CODE: ASMBLY	UNLESS OTHERWISE SPECIFIED DIMENSIONS ARE IN INCHES. TOLERANCES AND SPECS. ARE: DECIMALS ANGLES X ± .1 X ± .5 XX ± .03 XXX ± .015 HOLE SIZE TOLERANCE: (+ .005 / - .000) BREAK EDGES .005 - .015 CMFR DRILLED HOLES .03 X 45°	ALTIMATE MEDICAL, INC. PO BOX 180 - 262 WEST FIRST STREET - MORTON, MN 56270 PHONE (507) 697-6393 FAX (507) 697-6900 DESCRIPTION: ASMBLY FOLLOWING ARM YOUTH SIZE DWG. NO. NG50436 REV. A SCALE = 1:3 CAD FILE NAME: NG50436 SHEET 1 OF 1
--	--	---	--	--