
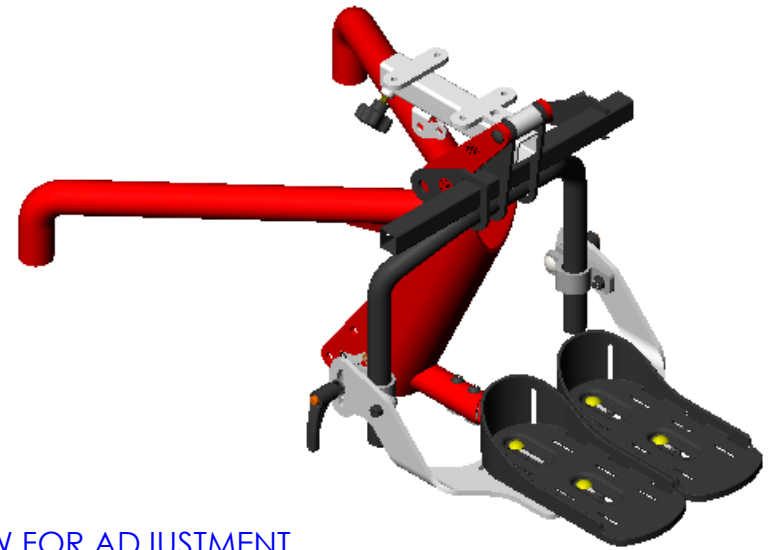


THE INFORMATION CONTAINED IN THIS DRAWING IS THE SOLE PROPERTY OF ALTIMATE MEDICAL. ANY REPRODUCTION IN PART OR WHOLE WITHOUT THE WRITTEN PERMISSION OF ALTIMATE MEDICAL IS PROHIBITED.

REVISIONS				
REV:	DESCRIPTION:	ECO #:	DATE:	APPROVED:
000	NEW RELEASE.	DCR-00460	10/15/08	MWL
A	REPLACED ITEM 600132 WITH 600436 AND 600126 WITH 600183.	DCR-00559	6/24/09	MWL
A	ADDED ITEM 600332.	DCR-00559	6/24/09	MWL
B	REVISED T20002 BY SHORTENING THE SIDE WALLS. REVISED TITLE.	DCR-00627	11/24/09	MWL

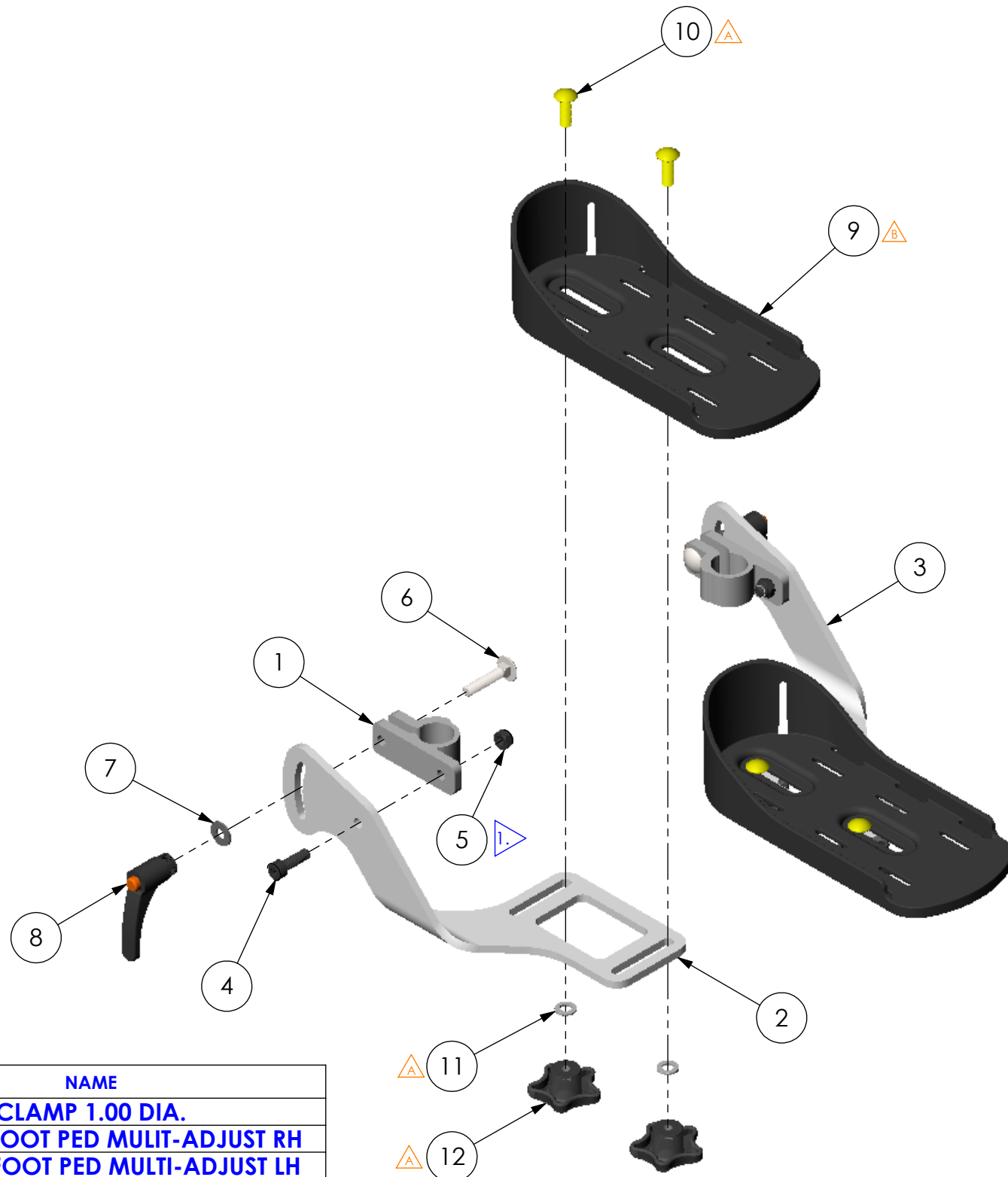

ITEM NO.	PART NO.	REV	QTY.	NAME
1	T10044	A	2	CLAMP 1.00 DIA.
2	T10047	000	1	BRACKET FOOT PED MULTI-ADJUST RH
3	T10048	000	1	BRACKET FOOT PED MULTI-ADJUST LH
4	600035	000	2	BOLT M8 X 1.25 - 25 SHCS
5	600009	000	2	NUT M8 X 1.25 NYLON LOCK
6	600266	000	2	BOLT CARRIAGE M8 X 1.25 - 40
7	600160	000	2	WASHER 1/4" USS ZINC
8	600092	000	2	M8 FEMALE QUICK ADJUST KNOB
9	T20002	A	2	FOOTPLATE SMALL
10	600183	000	4	BOLT CARRIAGE M8 X 1.25 - 25
11	600332	000	4	WASHER M8 ZINC
12	600436	000	4	KNOB FIVE POINT FEMALE M8 NYLOCK

NOTE:
1. LEAVE NUT JUST LOOSE ENOUGH TO ALLOW FOR ADJUSTMENT.

IF BOX CONTAINS AN "X", SEE NOTES THAT APPLY TO EACH SUBJECT.		APPROVED: _____ DATE: _____		UNLESS OTHERWISE SPECIFIED DIMENSIONS ARE IN INCHES. TOLERANCES AND SPECS. ARE: DECIMALS ANGLES .X ± .1 .X ± .5° .XX ± .03 .XXX ± .015 HOLE SIZE TOLERANCE: (+.005/-.000) BREAK EDGES .005 - .015 CMFR DRILLED HOLES .03 X 45°	ALTIMATE MEDICAL, INC. PO BOX 180 - 262 WEST FIRST STREET - MORTON, MN 56270 PHONE (507) 697-6393 FAX (507) 697-6900 DESCRIPTION: ASMBLY FOOTPLATE MULTI-ADJUST SM SIZE DWG. NO. T50006 REV. B SCALE = 1:4 CAD FILE NAME: T50006 SHEET 1 OF 1
<input type="checkbox"/> SEE SHIPPING INSTRUCTIONS	<input type="checkbox"/> RECEIVING INSPECTION	DRAWN: MWL	DATE: 1/25/08		
<input type="checkbox"/> SEE ENVIRONMENTAL INSTRUCTIONS		MATERIAL:	FINISH: -		
		CODE: ASMBLY	OPTION NUMBER: -		