


THE INFORMATION CONTAINED IN THIS DRAWING IS THE SOLE PROPERTY OF
 ALTIMATE MEDICAL. ANY REPRODUCTION IN PART OR WHOLE WITHOUT
 THE WRITTEN PERMISSION OF ALTIMATE MEDICAL IS PROHIBITED.

REVISIONS				
REV:	DESCRIPTION:	ECO #:	DATE:	APPROVED:
000	NEW RELEASE.	DCR-00460	10/13/08	MWL

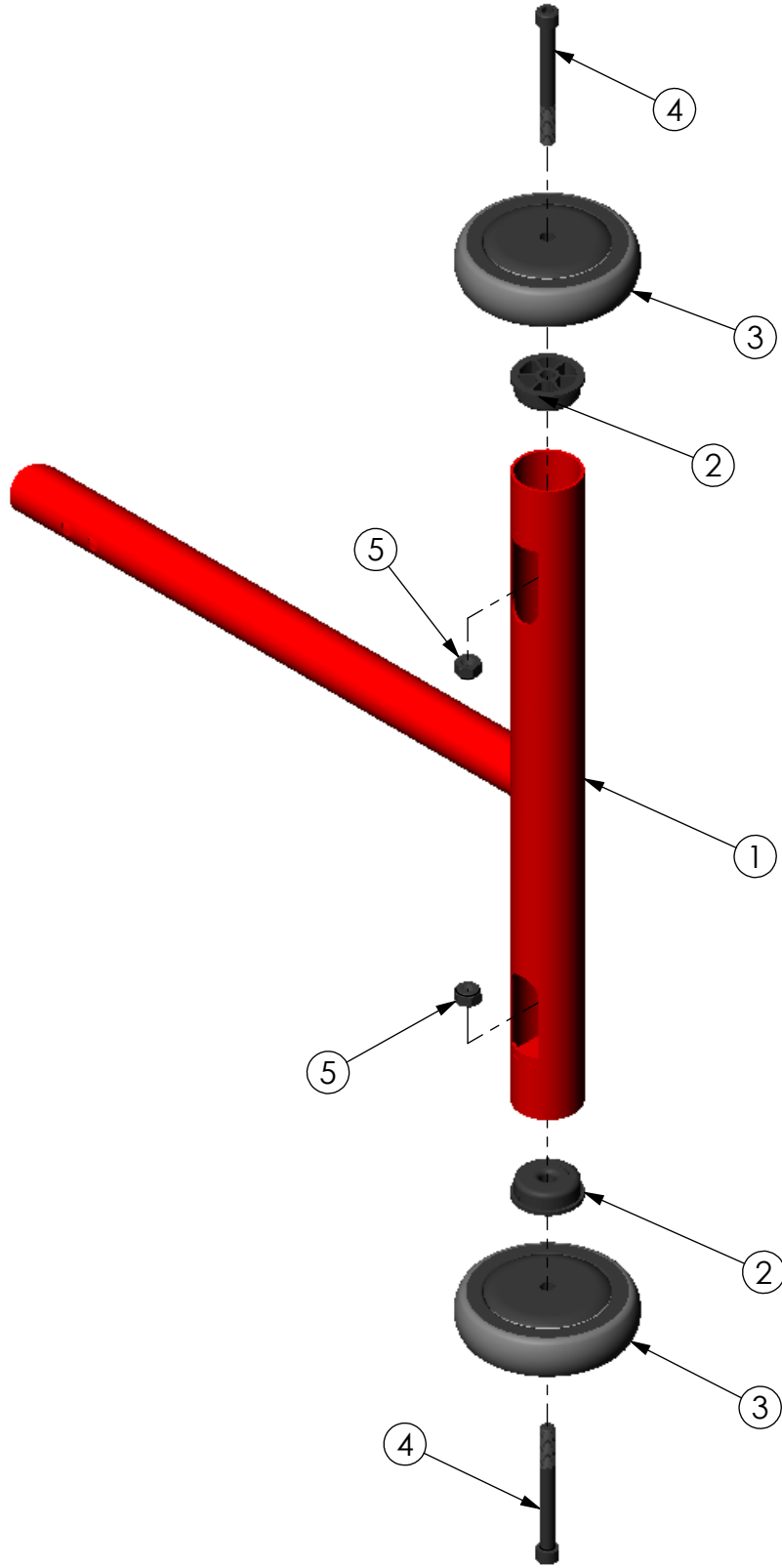

ITEM NO.	PART NO.	REV	QTY.	NAME
1	T10082	000	1	FRAME FRONT NO-TABLE PED
2	T20005	000	2	BUSHING WHEEL SUPPORT
3	600006	A	2	WHEEL FRONT 125mm DIA
4	600356	000	2	BOLT M10 X 1.5 - 100 SHCS
5	600010	000	2	NUT M10 X 1.5 NYLON LOCK

- IF BOX CONTAINS AN "X", SEE NOTES THAT APPLY TO EACH SUBJECT.
- SEE SHIPPING INSTRUCTIONS
 - RECEIVING INSPECTION
 - SEE ENVIRONMENTAL INSTRUCTIONS

APPROVED:	DATE:
DRAWN: MWL	DATE: 1/30/08
MATERIAL:	FINISH:
OPTION NUMBER:	CODE: ASMBLY

UNLESS OTHERWISE SPECIFIED
 DIMENSIONS ARE IN INCHES.
 TOLERANCES AND SPECS. ARE:

DECIMALS	ANGLES
.X ± .1	.X ± .5°
.XX ± .03	
.XXX ± .015	
HOLE SIZE TOLERANCE: (+.005/- .000)	
BREAK EDGES .005 - .015	
CMFR DRILLED HOLES .03 X 45°	

ALTIMATE MEDICAL, INC.
 PO BOX 180 - 262 WEST FIRST STREET - MORTON, MN 56270
 PHONE (507) 697-6393 FAX (507) 697-6900

DESCRIPTION:
 ASMBLY NO-TABLE FRONT

SIZE DWG. NO. **T50016** REV. 000

SCALE = 1:5 CAD FILE NAME: T50016 SHEET 1 OF 1