


THE INFORMATION CONTAINED IN THIS DRAWING IS THE SOLE PROPERTY OF
 ALTIMATE MEDICAL. ANY REPRODUCTION IN PART OR WHOLE WITHOUT
 THE WRITTEN PERMISSION OF ALTIMATE MEDICAL IS PROHIBITED.

REVISIONS				
REV:	DESCRIPTION:	ECO #:	DATE:	APPROVED:
000	NEW RELEASE.	DCR-01002	11/27/12	MWL

ITEM NO.	PART NO.	REV	QTY.	NAME
1	Y1004	000	1	AB BRACKET BANTAM MED
2	Y2004	000	2	HIP PIVOT BUSHING
3	Y1005	000	1	HIP BRACKET BANTAM MED
4	600087	000	2	BOLT M12 X 20 SHOULDER
5	600325	000	2	WASHER FLAT M12
6	600010		2	NUT NYLOCK M10 X 1.5
7	Y2005	000	2	HIP COVER
8	600547		6	BOLT FLT SHCS M5 X .8 - 10
9	80825	000	1	SNAP BUTTON
10	600370	000	1	PIN ROLL 3mm DIA X 24mm LONG
11	600500	000	2	BEARING R8PPX
12	Y2006	000	4	COVED WASHER 1.00 DIA
13	600159	000	4	WASHER NYLON M8
14	600009		9	NUT NYLOCK M8 X 1.25
15	Y4000	000	2	LABEL FOLLOWING ARM DEPTH
16	600565	000	2	BOLT SHCS M6 X 1.0 - 6
17	Y4001	000	6	LABEL HAND CRUSH
18	Y1006	000	2	FOLLOWING ARM OUTER BANTAM MED
19	600524	000	2	TOGGLE LOCK KNOB
20	600525	000	2	RETAINING RING 14mm
21	600527	000	2	PIN CLEVIS 5/16" SHOULDERED
22	600528	000	2	E-CLIP 5/16"
23	Y1072	000	1	BRACKET BACK REST MOUNT RH
24	Y1073	000	1	BRACKET BACK REST MOUNT LH
25	Y2008	000	2	BUSHING BACK REST BRKT
26	600460	00	2	BOLT SHOULDER M10 X 30
27	Y1021	000	1	BACK ANGLE TUBE INNER
28	Y2027	000	2	COVED WASHER 1.125 DIA
29	600352	000	3	BOLT SHOULDER M10 X 50
30	Y1020	000	1	ANGLE ADJUST TUBE OUTER
31	80283	01	1	FLAT WASHER
32	81256	000	1	WASHER-SHOULDER, NYLON
33	Y1023	000	1	SHAFT ADJ. BACK REST
34	600160	000	2	WASHER 1/4" USS ZINC
35	600334	000	1	WASHER LOCK 8mm INTERNAL TOOTH
36	600412		1	BOLT BTN SHCS M8 X 1.25 - 16
37	600011		1	NUT NYLOCK M12 X 1.75
38	NG20122	A	1	KNOB FEMALE M12 X 1.75
39	600147	000	4	WASHER NYLON M10
40	600549	000	2	BOLT M10 X 12 SHOULDER
41	600548	000	1	CAP RND 1-1/8 10-14GA
42	Y2046	000	2	COVER FOLLOWING ARM OUTER
43	Y2047	000	2	COVER FOLLOWING ARM INNER
44	Y2048	000	2	BOLT HEAD CAP
45	600213	000	2	BOLT SHOULDER M10 X 45
46	600589	000	1	LANYARD SEAT DEPTH STOP
47	600454		2	BOLT BTN SHCS M5 X 0.8 - 10
48	Y1133	000	2	FOLLOWING ARM INNER BANTAM MED
49	600597	000	2	PIN SPRING 4mm X 24mm

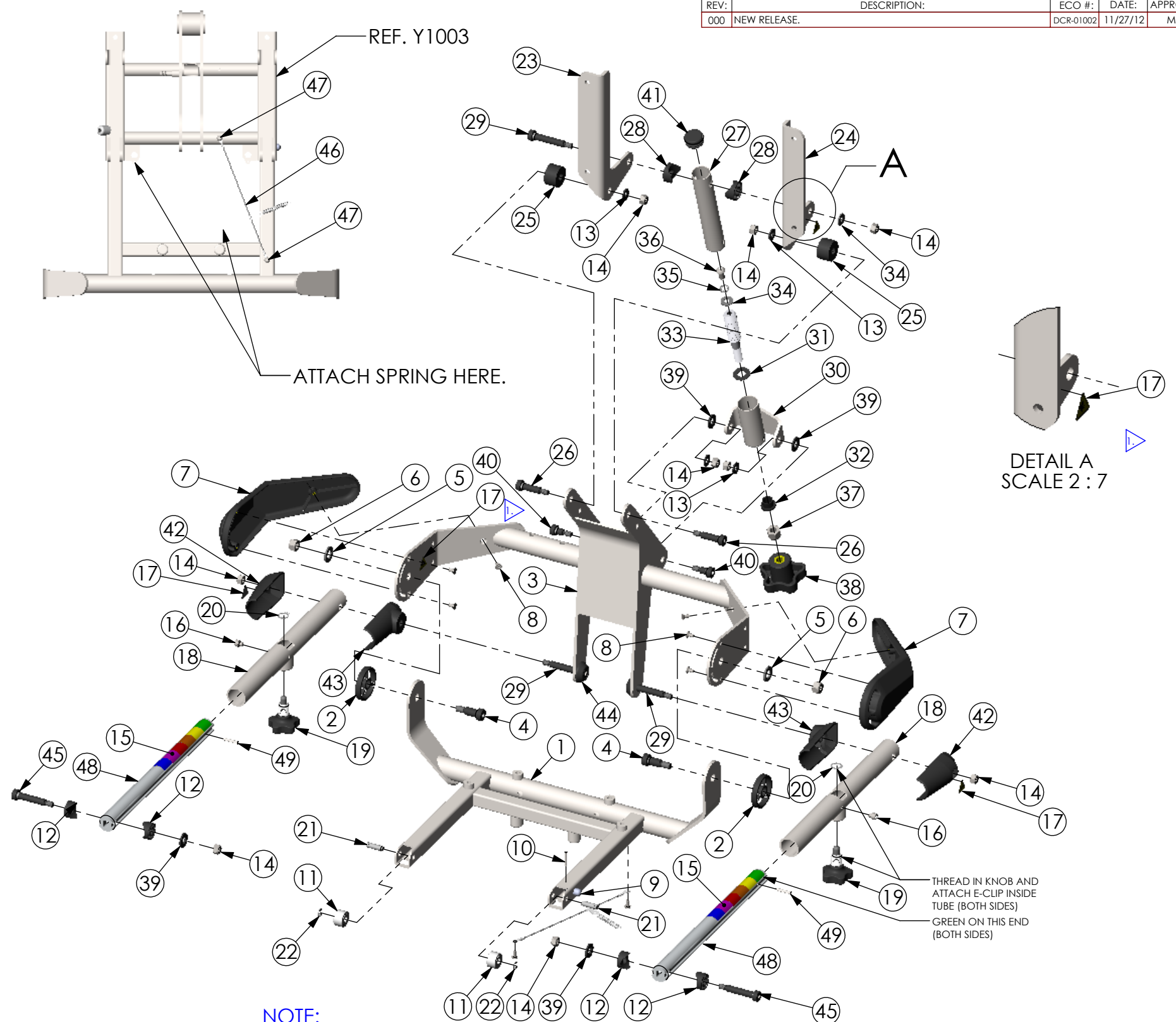

NOTE:
 1. PLACE DECAL IN SAME WAY ON BOTH SIDES OF UNIT.

IF BOX CONTAINS AN "X", SEE NOTES THAT APPLY TO EACH SUBJECT.	APPROVED:	DATE:	UNLESS OTHERWISE SPECIFIED DIMENSIONS ARE IN INCHES. TOLERANCES AND SPECS. ARE: DECIMALS .X ± .1 .XX ± .03 .XXX ± .015 HOLE SIZE TOLERANCE: (+.005/-.000) BREAK EDGES .005 - .015 CMFR DRILLED HOLES .03 X 45°	ALTIMATE MEDICAL, INC. PO BOX 180 - 262 WEST FIRST STREET - MORTON, MN 56270 PHONE (507) 697-6393 FAX (507) 697-6900 DESCRIPTION: ASMBLYBACK BANTAM MED SIZE DWG. NO. Y5528 REV. 000 SCALE = 1:7 CAD FILE NAME: Y5528 SHEET 1 OF 1	
<input type="checkbox"/> SEE SHIPPING INSTRUCTIONS	DRAWN: MWL	DATE: 7/20/11			MATERIAL:
<input type="checkbox"/> RECEIVING INSPECTION	EXPIRATION DATE:				FINISH:
<input type="checkbox"/> SEE ENVIRONMENTAL INSTRUCTIONS					CODE: ASMBLY