


THE INFORMATION CONTAINED IN THIS DRAWING IS THE SOLE PROPERTY OF  
 ALTIMATE MEDICAL. ANY REPRODUCTION IN PART OR WHOLE WITHOUT  
 THE WRITTEN PERMISSION OF ALTIMATE MEDICAL IS PROHIBITED.

| REVISIONS |              |           |          |           |
|-----------|--------------|-----------|----------|-----------|
| REV:      | DESCRIPTION: | ECO #:    | DATE:    | APPROVED: |
| 000       | NEW RELEASE. | DCR-01002 | 11/19/12 | MWL       |

D  
C  
B  
A

D  
C  
B  
A

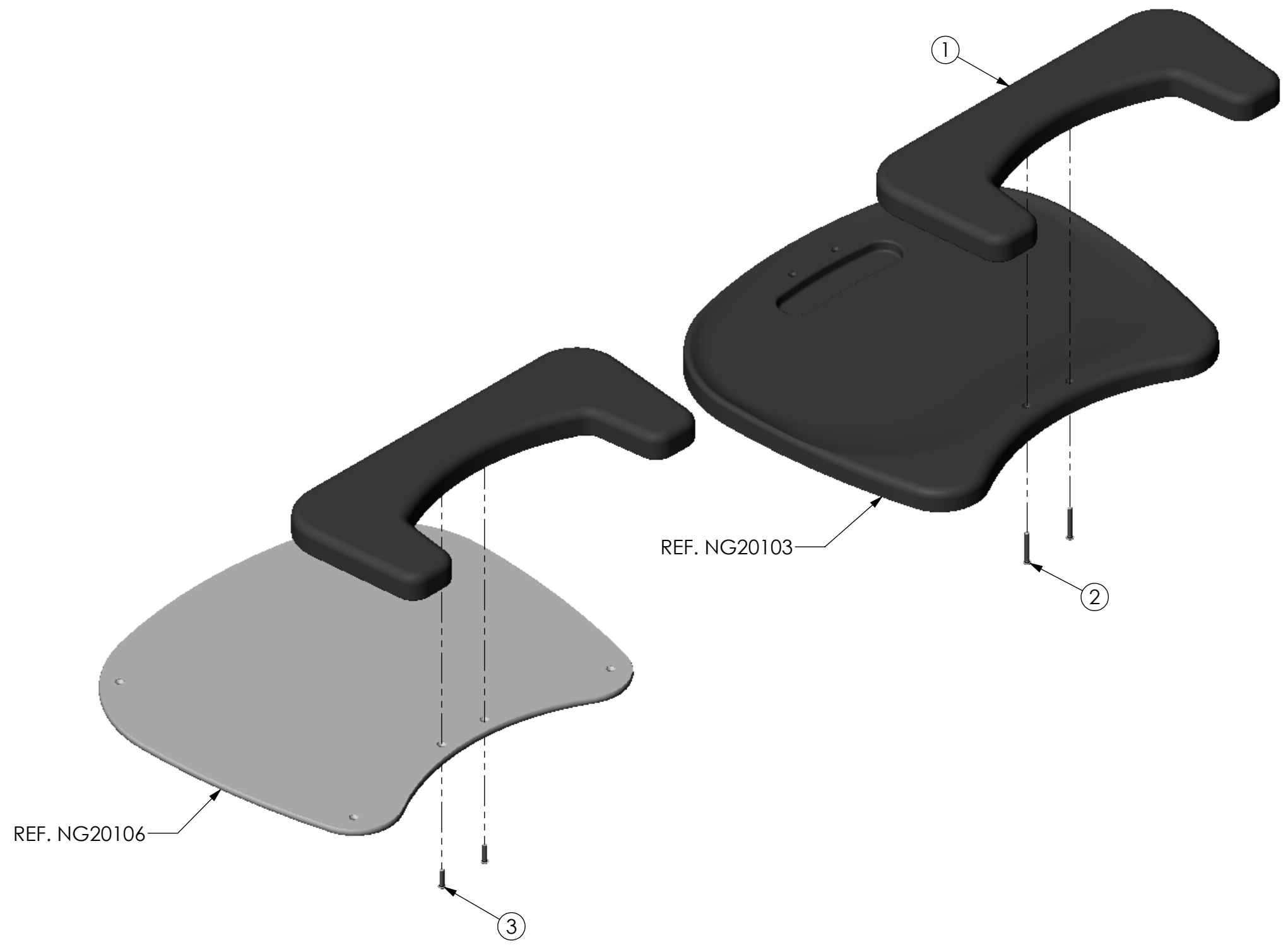

| ITEM NO. | PART NO. | REV | QTY. | NAME                        |
|----------|----------|-----|------|-----------------------------|
| 1        | Y3015    | 000 | 1    | ELBOW PAD MED BANTAM 13"    |
| 2        | 600472   |     | 2    | BOLT BTN SHCS M6 X 1.0 - 45 |
| 3        | 600473   |     | 2    | BOLT BTN SHCS M6 X 1.0 - 25 |

|   |                    |               |   |   |              |
|---|--------------------|---------------|---|---|--------------|
| IF BOX CONTAINS AN "X", SEE NOTES THAT APPLY TO EACH SUBJECT.<br><input type="checkbox"/> SEE SHIPPING INSTRUCTIONS<br><input type="checkbox"/> RECEIVING INSPECTION<br><input type="checkbox"/> SEE ENVIRONMENTAL INSTRUCTIONS | APPROVED:          | DATE:         | <b>UNLESS OTHERWISE SPECIFIED DIMENSIONS ARE IN INCHES. TOLERANCES AND SPECS. ARE:</b><br>DECIMALS: .X ± .1<br>.XX ± .03<br>.XXX ± .015<br>ANGLES: .X ± .5°<br>HOLE SIZE TOLERANCE: (+.005/- .000)<br>BREAK EDGES .005 - .015<br>CMFR DRILLED HOLES .03 X 45° | <b>ALTIMATE MEDICAL, INC.</b><br>PO BOX 180 - 262 WEST FIRST STREET - MORTON, MN 56270<br>PHONE (507) 697-6393 FAX (507) 697-6900<br>DESCRIPTION:<br><b>ASSEMBLY ELBOW PAD 13"</b><br>SIZE DWG. NO. <b>Y5582</b> REV. <b>000</b><br>SCALE = 1:6 CAD FILE NAME: Y5582 SHEET 1 OF 1 |              |
|   | DRAWN: MWL         | DATE: 5/30/12 |   |   | MATERIAL:    |
|   | EXPIRATION DATE: - | FINISH: -     |   |   | CODE: ASMBLY |
|   |                    |               |   |   |              |

8 7 6 5 4 3 2 1