

EasyStandⁱⁱ Bantam

PT50001-1 Extra Small and PT50002-1 Small

Assembly Manual



Please ensure the Assembly Manual stays with the unit at all times.

Table of Contents

Assembly 1


Mobile Assembly10

IMPORTANT
EasyStand Warranty Registration
FREE GIFT Upon Completion
(warranty & offer valid in the United States only)

IMPORTANT! Please complete and return within 30 days. To receive the free gift, please fill out all information. Thank You.

To register online go to easystand.com/warranty

**or Scan the
QR Code**

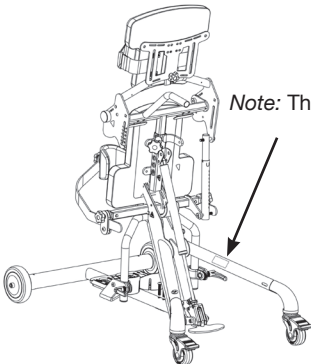

Note: The serial number is located on the leg bracket.

Date Purchased: _____


Serial #: _____

Contact Altimate Medical for any information required prior to purchase. This owner’s manual along with other information is freely available by request or by downloading from easystand.com.

SYMBOL/LABEL LEGEND



CAUTION/WARNING/ATTENTION

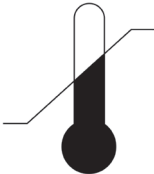

TEMPERATURE



TYPE BF APPLIED PART



DATE OF MANUFACTURE



PINCHPOINT



SEE INSTRUCTIONS FOR USE



HUMIDITY



ATMOSPHERIC PRESSURE

SN

SERIAL NUMBER



MANUFACTURER

CE

CONFORMITÉ EUROPÉENNE

R_xonly

PRESCRIPTION ONLY

MD

MEDICAL DEVICE



POTENTIALLY HOT SURFACE

Accessible surfaces which may exceed 105°F/41°C and could be a hazard to persons with skin damage or not having sensation.




Waste of electrical and electronic equipment must not be disposed as unsorted municipal waste. It must be collected separately, and must be disposed as per local regulations. Contact your authorized representative for information concerning the decommissioning of your equipment.




CERTIFICATION LABEL/UNIQUE DEVICE IDENTIFICATION LABEL

Serial Number


Item Number
Item Description




Manufacturer:
Altimate Medical, Inc
262 West First Street
Morton, MN 56270
507-697-6393



2021




SN 0000000000



00000000000000000000

Model:
Height Range:
Max. Weight Limit:

Approx. Max. Mass of Unit
& Weight Limit: 0000



Unique Identifier

Assembly


1. Remove all parts from the box. Remove the two bolts and two covered nuts from front frame.



2. Flip up the rear frame to allow easier access to the post. Attach the front and rear frames to one another.



3. Place the two covered nuts into the bottom of the frame and flush with the tube. You may have to tap them in with a rubber hammer to make them flush.


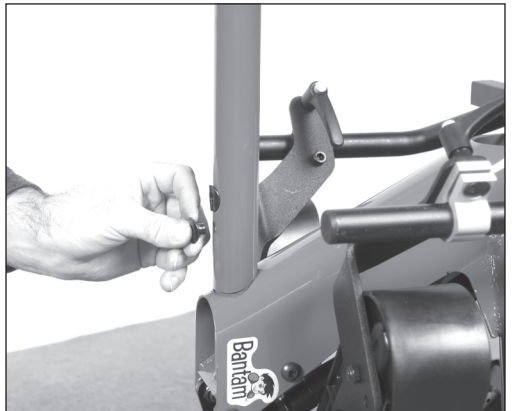


4. Place the two bolts into the top of the frame and screw into the coved nut, making sure they are flush. Tighten securely.



5A. Place the unit back in the normal position. If you ordered the optional Manual Hydraulic Actuator with Removable Handle, remove knob, bolt and spacer from center of handle. Insert spacer as shown.

5B. Place bolt into the spacer as shown.

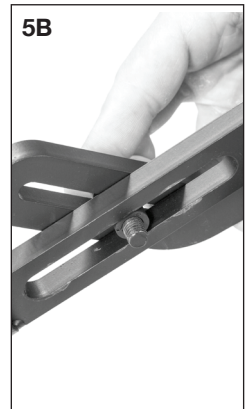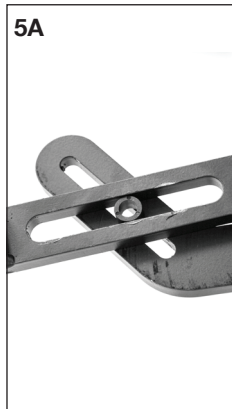

6. Place knob onto bolt and tighten securely.



7. Remove pin from the pump, place pump handle onto pump shaft as shown and place pin through holes.




8. Place top of handle on to the handle, insert 2 bolts and tighten securely.



9A. If the Gas Spring Lockout was ordered, unlock the gas spring by pulling the plunger.

9B. If the Gas Spring Lockout was not ordered, remove the bolt from the lower part of the cylinder on the mast as shown, discard bolt.

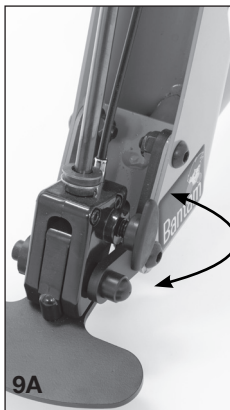

9. Use the optional pump handle or step on the gas spring assist pedal to raise the seat about 3/4 the way up as shown. Line up the following arm with the back angle tube.



10. Depress the spring button to allow the following arm to slide into place. Match the following arm position color with the corresponding seat depth color. Tighten the knob securely (for information on seat adjustments see the Instructions for Use Manual).



11. Loosen the ratchet handles on the foot plates and swivel into place (for information on foot plate adjustments see the Instructions for Use Manual).



12A. Insert each knee bracket into the tubes onto the unit, with the clamp down.

12B. Depress the spring button to allow the bracket to slide into place, advance until the spring button locks into place.

For the Swing-Away Knee Pads, insert bracket into place and tighten knobs securely.



13. Insert each knee pad bracket into the slots on the knee bracket. Tighten knobs securely.




14. Remove knob from the knee pad, leaving the plastic and rubber washer in place on the knee pad, place the knee pad and washers onto the knee pad bracket. Place knob onto bolt as shown. Repeat steps for the opposite side (for information on knee pad adjustments see the Instructions for Use Manual).



15. If you ordered the Supine option, remove the controller from the packaging. Remove the two bolts on the bottom of the controller, leaving nuts in place.




16. Place controller onto the bracket on the side of the unit. The bracket will slide into the slot on the bottom of the controller.

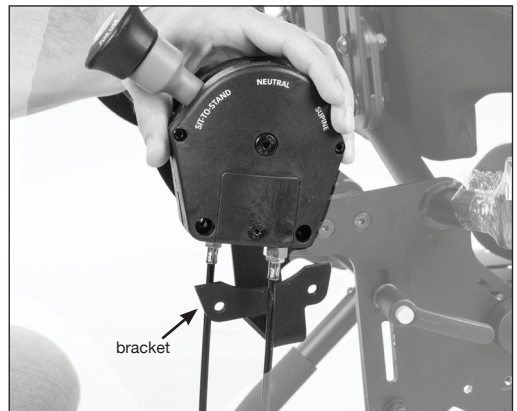

17. Making sure the nuts are in the back on the controller, line up holes and insert both bolts. Tighten securely.




18. If you ordered the 11"x19" Black molded tray, place tray on the floor upside down as shown. The longer tray arm will go on the right side, the short arm on the left side.

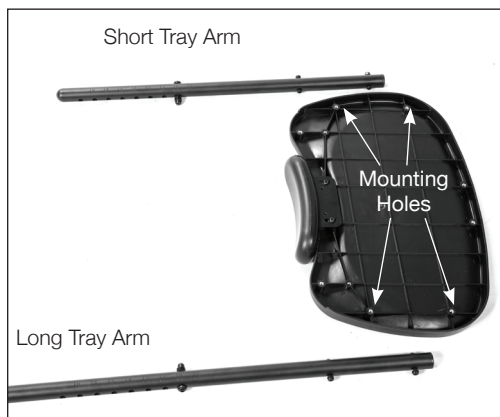

19A. Remove the four nuts from the mounting bolts on the arms and discard, do not remove the acorn nut and bolt.


19B. With the acorn nut facing up, place the longer arm on the right side of the tray, line up the two mounting holes. Insert bolt, and tighten snug.

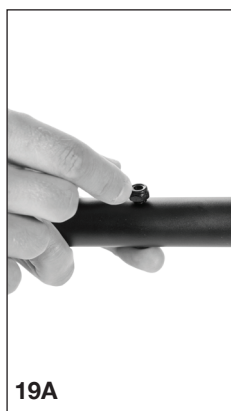

20A. Place the second bolt into the hole.



20B. Using an allen wrench tighten both bolts securely.

With the acorn nut facing up, place the shorter arm on the left side of the tray and repeat steps 18-19.

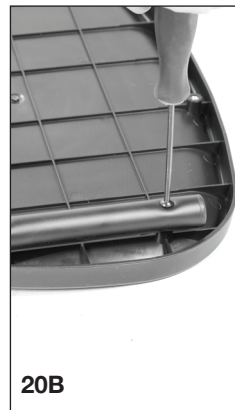

21A. For the tray, remove the nut from the bolt on the tray arm and discard the nut. Make sure all other hardware stays on the bolt.

21B. Place the ratchet handle onto the bolt and tighten securely. Repeat steps for the other side.

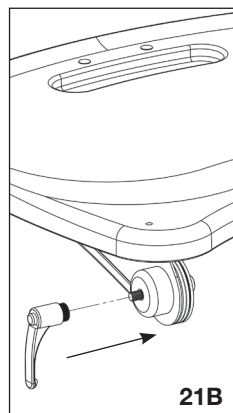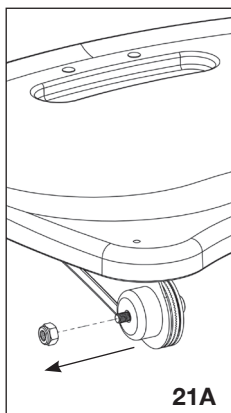

22. If you ordered the optional Shadow Tray, loosen the ratchet handles on the side of the tray, flip the arms up so they are lined up with the tray as shown and tighten the ratchet handles. Insert the two tray arms into the tray bracket on the bottom of the tray. One of the arms is longer than the other for easier installation (for information on Shadow Tray adjustments see the Instructions for Use Manual). The bolts on the arms are used as a stop for future reference. Place the quick release pins into the holes of the tray to secure into place.



23. For the Swing-Away Shadow Tray, the knobs are located in the bag your manual came in. Place knobs onto unit as shown, insert tray bracket and tighten securely.





Caution: After placing the tray into position, ensure the ratchet handles on the side of the tray are tightened securely.




24A. If not already on the unit, remove the knob from the center post and place the back onto the unit as shown.

24B. Place the back in the desired position. Insert knob and tighten securely.

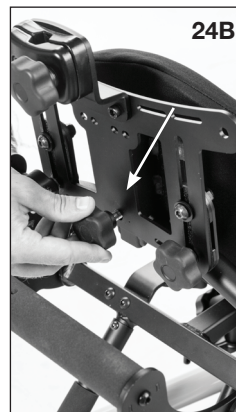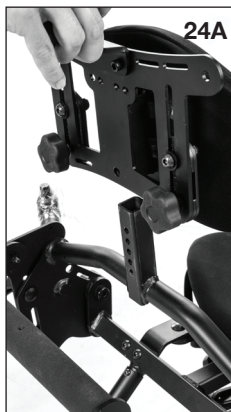


25A. If the optional dual lift control was ordered, insert bolt into push handle as shown and line up dual controller. Screw the bolt into the dual controller, tighten securely.

25B. If you ordered the optional headrest, insert the head support onto the unit and tighten securely.

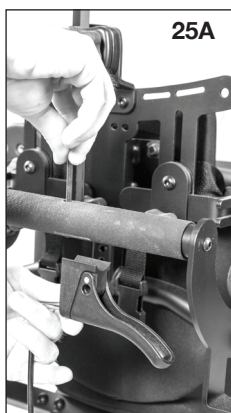

26A. If you ordered the optional Elbow Pad Cutout, remove Chest Pad from tray. Disregard bolts, replace bolts with bolts in the bag with the Elbow Pad Cutout. Place Elbow Pad Cutout onto the tray. Line up Chest Pad bracket with holes, insert bolts into Tray and Elbow Pad Cutout. Tighten securely.

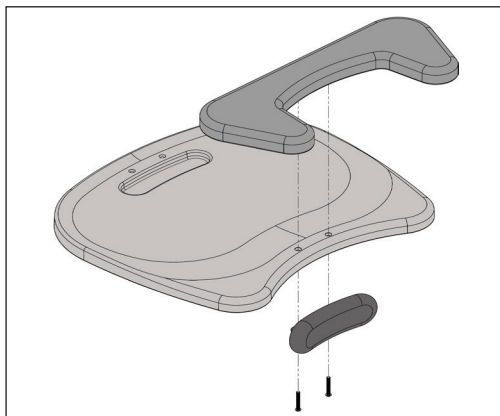

Mobile Option Assembly

Note: If this is an option added after the initial purchase, you must remove the existing front assembly before proceeding.

27. The Mobile option consists of two drive wheel assemblies, a “T” style front weldment, barrel nuts and bolts. You will need a 5mm allen wrench and a rubber hammer for assembly.




28. Set the Bantam on its back and attach the “T” weldment to the frame. Note that the front curves upward.



29. Line up holes and place the covered nuts on the bottom of the frame. You may have to tap them with a rubber hammer to make them flush. Tighten bolts securely.



30. Attach the interchangeable right and left drive wheel assemblies to the front frame.

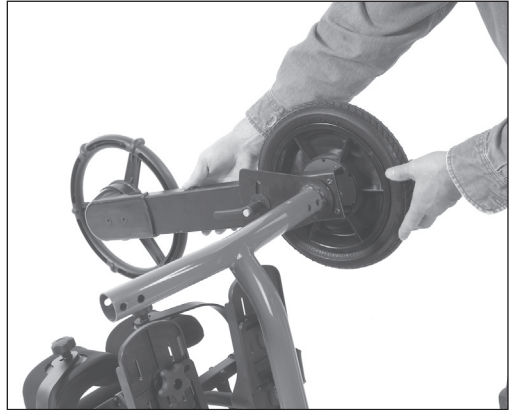


31. Line up the holes and tap in the covered nuts and tighten bolts securely. Making sure the covered nuts are flush with the base. Position the Bantam upright and re-tighten the frame bolts.




Mobile Adjustment & Usage

32. To lock or unlock the drive wheels, turn the drive wheel lock one quarter turn.

Use the push rim adjustment to position the push rims forward or aft, for transferring or self-propulsion. Tighten to lock in position.

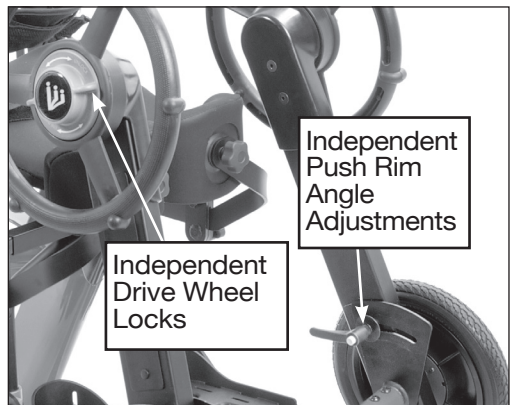

Options & Parts Detail

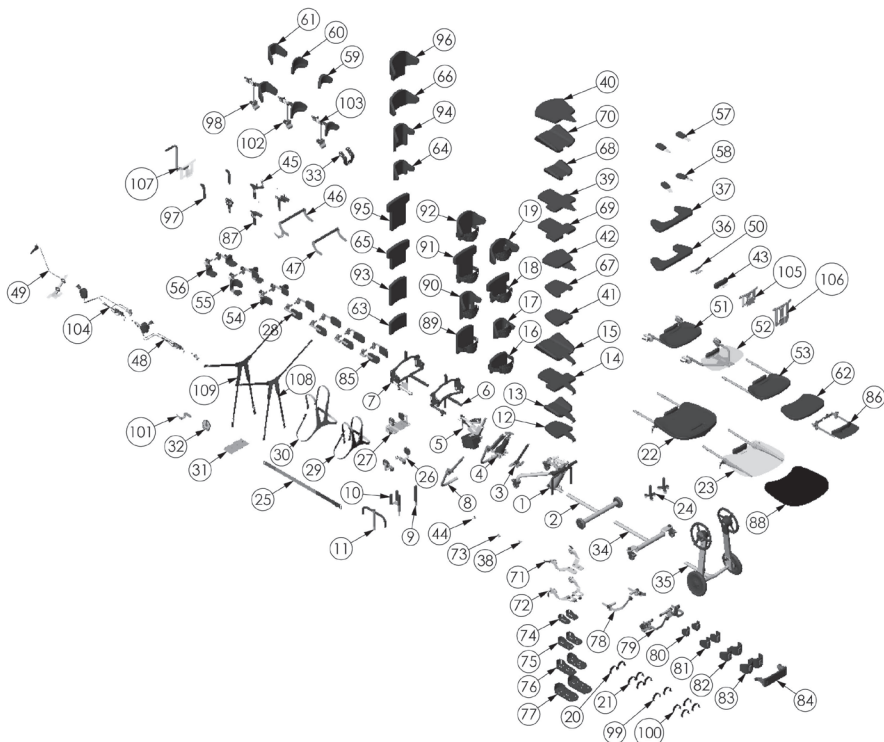

ITEM NO.	PART NO.	NAME	ITEM NO.	PART NO.	NAME
1	150000	ASMBLY FRAME PEDIATRIC	69	130084	SLIP COVER SEAT STANDARD SM
2	150016	ASMBLY GAS SPRING PEDIATRIC	70	130084	SLIP COVER SEAT COMFY SM
3	150018	ASMBLY HYDRAULIC LIFT PEDIATRIC	71	150250	FOOT PLATE STANDARD
4	150020	ASMBLY TRAY SWING AWA L MPS	72	150252	FOOT PLATE MULTI-ADJUST
5	150022	ASMBLY BACK REST BANTAM	73	150252	NON SUPINE ASSEMBLY
6	150100	BACK REST ASMBLY XS	74	150254	FOOT HOLDERS 11 7/8" X 3" W
7	150102	ASMBLY BACK REST SMALL	75	150256	FOOT HOLDERS 7 7/8" X 3 25" W
8	150102	HANDLE PUMP REMOVABLE/ADJ	76	150256	FOOT HOLDERS 11 7/8" X 5" W
9	150220	HANDLE STANDARD TOP	77	150260	KNEE STANDARD BANTAM
10	150044	ACTUATOR QUAD GRIP OPTION	78	150262	SWING-AWAY KNEES BANTAM
11	150044	ASMBLY SEAT STANDARD XS	79	150266	KNEE PAD 3 25"
12	150042	ASMBLY SEAT FORM TO FIT XS	80	150266	KNEE PADS 4 25"
13	150042	ASMBLY SEAT STANDARD SM	81	150272	KNEE PADS 4 25"
14	150048	ASMBLY UPHOLSTRY COMFY SEAT SM	82	150272	ASMBLY KNEE PAD SOFT MPS
15	150050	ASMBLY BACK REST STANDARD CKXS	83	150276	ASMBLY LATERAL GUIDES PED
16	150052	ASMBLY BACK REST STANDARD BCKXS	84	150276	MOBILE TABLE BANTAM
17	150054	ASMBLY UPHOLSTRY COMFY BACK SM	85	150276	ASMBLY SHADOW SUPPORT TRAY PED
18	150056	ASMBLY FOOT STRAP SECURE PED	86	150276	PADDED SLIP COVER LARGE TRAY
19	150058	ASMBLY TABLE CLEAR PEDIATRIC	87	150276	BACK REST HIGH 15" H
20	150060	ASMBLY TABLE CLEAR PEDIATRIC	88	150276	ASMBLY UPHOLSTRY FORM TO FIT BACK HIGH XS
21	150062	SUPPORT GRIP OPTION	89	150276	ASMBLY BACK REST STANDARD HIGH SM
22	150064	POSITIONING BELT ADJUSTABLE 44.5" - 44.5"	90	150276	ASMBLY UPHOLSTRY COMFY BACK SM
23	150066	ASMBLY HIP GUIDES PEDIATRIC	91	150276	SLIP COVER BACK HIGH XS
24	150068	ASMBLY HIP GUIDES XS	92	150276	SLIP COVER BACK HIGH SM
25	150070	ASMBLY LATERAL GUIDES PED	93	150276	SLIP COVER BACK COMFY HIGH SM
26	150072	X-STYLE CHEST VEST 11" X 9.5" W	94	150276	NO TRAY HANDLE SUPPORT
27	150074	X-STYLE CHEST VEST 11" X 9.5" W	95	150284	ASMBLY HEAD REST FORM TO FIT
28	150076	TOE POST	96	150284	ASMBLY FOOT STRAP SECURE 15" L
29	150078	SEAT ANGLE ADJUSTOR	97	150284	BRACKET CONTROLLER
30	150080	ASMBLY HIGH MOUNT CHEST VEST BRKT	98	150284	ASMBLY HEAD REST SM
31	150082	ASMBLY SWIVEL CASTER BANTAM	99	150284	SUPINE OPTION W/MECH LOCK BYPASS
32	150084	ASMBLY ELBOW PAD 7 1/2" CUTOOUT	100	150284	ASMBLY BACK REST BRACKET LOW
33	150086	ASMBLY ELBOW PAD 7 1/2" CUTOOUT	101	150284	NON-ASSIST HANDLE SMALL BANTAM
34	150088	ASMBLY ELBOW PAD 7 1/2" CUTOOUT	102	150284	NON-STRETCH X-STYLE CHEST VEST 11" X 9.5" W
35	150090	UPHOLSTRY SEAT STANDARD SM	103	150284	ASMBLY LATERAL GUIDES PED
36	150092	UPHOLSTRY SEAT STANDARD SM	104	150276	ASMBLY LATERAL GUIDES PED
37	150094	UPHOLSTRY SEAT COMFY XS	105	150276	
38	150096	ASMBLY GAS SPRING BANTAM	106	150276	
39	150098	ASMBLY SHADOW SUPPORT TRAY PED	107	150276	
40	150100	HANDLE ASSEMBLY NON-SUPINE	108	150276	
41	150102	SUPINE OPTION SMALL BANTAM	109	150276	
42	150104	ASMBLY CHEST PAD MOUNT STANDARD	110	150276	
43	150106	ASMBLY TRAY SWING AWA L MPS	111	150276	
44	150108	ASMBLY TRAY SWING AWA L MPS			
45	150110	TABLE STANDARD 11 7/8" X 3 25" W			
46	150112	LATERAL ELBOW STOP 8" - 12"			
47	150114	LATERAL ELBOW STOP 8" - 12"			
48	150116	ASMBLY FOREARM WINGS MPS			
49	150118	ASMBLY FOREARM WINGS MPS			
50	150120	SLIP COVER HEAD REST 10" W			
51	150122	SLIP COVER HEAD REST 10" W			
52	150124	PADDED SLIP COVER TRAY MPS			
53	150126	SLIP COVER BACK COMFY 15" - 17" H			
54	150128	SLIP COVER BACK STANDARD 15" - 17" H			
55	150130	SLIP COVER BACK COMFY SM			
56	150132	SLIP COVER SEAT COMFY XS			
57	150134	SLIP COVER SEAT COMFY XS			
58	150136	SLIP COVER SEAT COMFY XS			
59	150138	SLIP COVER SEAT COMFY XS			
60	150140	SLIP COVER SEAT COMFY XS			
61	150142	SLIP COVER SEAT COMFY XS			
62	150144	SLIP COVER SEAT COMFY XS			
63	150146	SLIP COVER SEAT COMFY XS			
64	150148	SLIP COVER SEAT COMFY XS			
65	150150	SLIP COVER SEAT COMFY XS			
66	150152	SLIP COVER SEAT COMFY XS			
67	150154	SLIP COVER SEAT COMFY XS			
68	150156	SLIP COVER SEAT COMFY XS			
69	150158	SLIP COVER SEAT COMFY XS			
70	150160	SLIP COVER SEAT COMFY XS			
71	150162	SLIP COVER SEAT COMFY XS			
72	150164	SLIP COVER SEAT COMFY XS			
73	150166	SLIP COVER SEAT COMFY XS			
74	150168	SLIP COVER SEAT COMFY XS			
75	150170	SLIP COVER SEAT COMFY XS			
76	150172	SLIP COVER SEAT COMFY XS			
77	150174	SLIP COVER SEAT COMFY XS			
78	150176	SLIP COVER SEAT COMFY XS			
79	150178	SLIP COVER SEAT COMFY XS			
80	150180	SLIP COVER SEAT COMFY XS			
81	150182	SLIP COVER SEAT COMFY XS			
82	150184	SLIP COVER SEAT COMFY XS			
83	150186	SLIP COVER SEAT COMFY XS			
84	150188	SLIP COVER SEAT COMFY XS			
85	150190	SLIP COVER SEAT COMFY XS			
86	150192	SLIP COVER SEAT COMFY XS			
87	150194	SLIP COVER SEAT COMFY XS			
88	150196	SLIP COVER SEAT COMFY XS			
89	150198	SLIP COVER SEAT COMFY XS			
90	150200	SLIP COVER SEAT COMFY XS			
91	150202	SLIP COVER SEAT COMFY XS			
92	150204	SLIP COVER SEAT COMFY XS			
93	150206	SLIP COVER SEAT COMFY XS			
94	150208	SLIP COVER SEAT COMFY XS			
95	150210	SLIP COVER SEAT COMFY XS			
96	150212	SLIP COVER SEAT COMFY XS			
97	150214	SLIP COVER SEAT COMFY XS			
98	150216	SLIP COVER SEAT COMFY XS			
99	150218	SLIP COVER SEAT COMFY XS			
100	150220	SLIP COVER SEAT COMFY XS			

AltimateMedical

Assembly Manual

262 West 1st St.
Morton, MN 56270 USA
Toll Free: 800.342.8968 or 507.697.6393
Fax Free: 877.342.8968 or 507.697.6900
email: info@easystand.com
easystand.com



Medical Product Service GmbH
Borngasse 20
35619 Braunfels
Deutschland



MPS Medical Product Service UK Ltd
Northgate House
Northgate Street
Devizes
Wiltshire England SN10 1JX



MPS Medical Product Service Switzerland, CO/Accumed Sagl
Morbio Inferiore
Viale Serfontana 10
CH-6834 Switzerland



Standing technology should only be used under the guidance of a physician with recommendations for standing program protocol and any medical precautions. Standing programs should be monitored by the attending therapist. AMI maintains a policy of continual product improvement and reserves the right to change features, specifications, and prices without prior notification. Check with AMI for latest info.
FORM MBANEX-SMALL 120723 REV A Copyright © 2023 Altimate Medical, Inc. All rights reserved. Printed in the U.S.A.